


# Short operating instructions

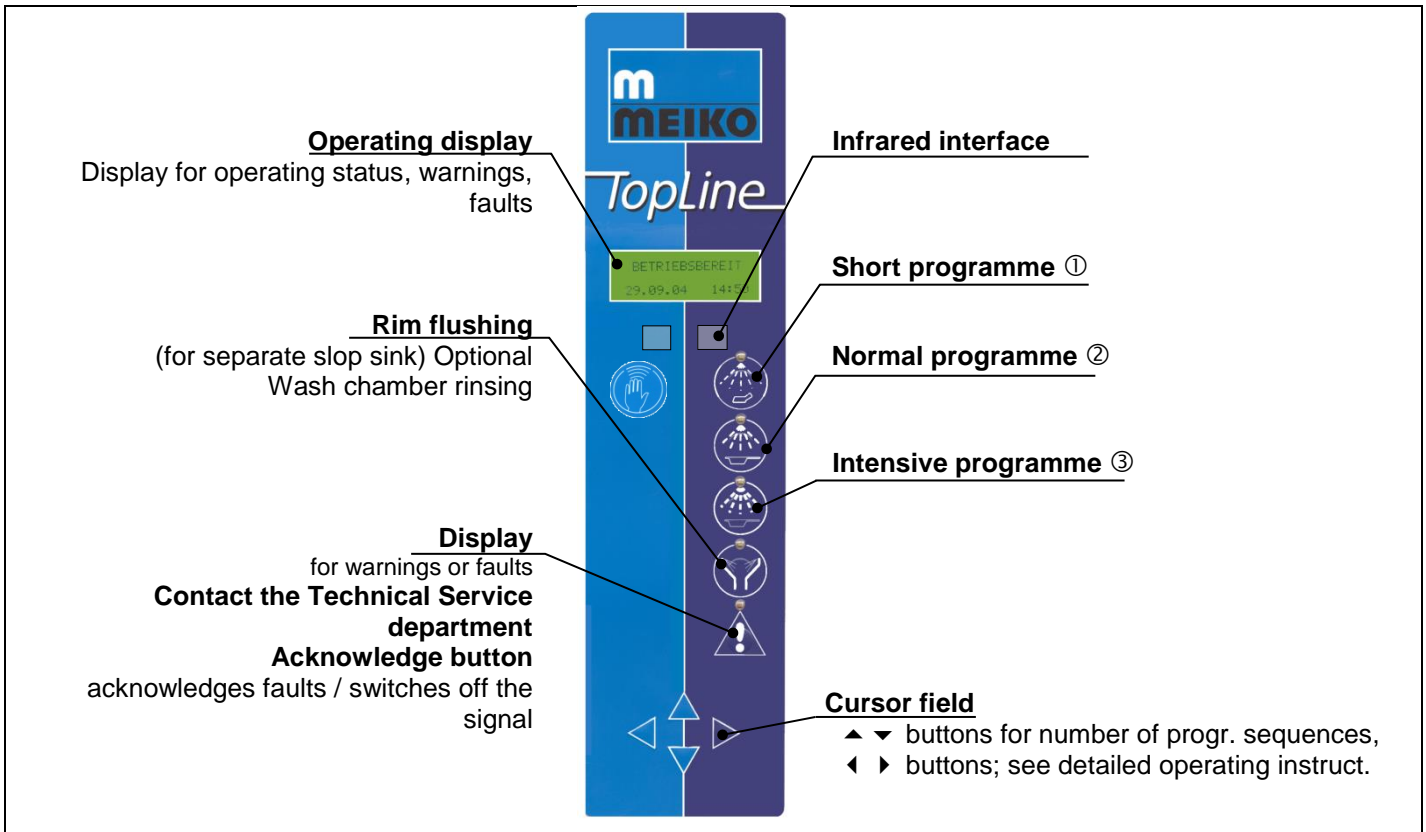

|  |  |  |  |   |
|--|--|--|--|---|
|  |  |  |  |   |
| <p>1. Open wash chamber door. Infrared or optional pedal switch.</p> |  | <p>2. Place the objects to be cleaned in the intended positions in the utensil holder.</p> |  | <p>3. Select the cleaning programme to reflect the degree of soiling and start the programme by lightly pressing the corresponding programme button. The door closes automatically.</p> |

|  |  |
|--|--|
|  |  |
| <p>4. At the end of the programme open the wash chamber door. Infrared or optional pedal switch.</p> |  |

**! Caution!**

**Crushing hazard**  
Door movement can crush fingers, hands and arms.

- Do not reach into the machine when the door is in motion.
- Contact the home automation or service department to remove any door blockages.