

# Operating instructions

## TopClean 60

### Multiwasher and disinfection machine

Translation of the "Original operating instructions"



EN



**Read operating instructions before using the Multiwasher!**

# **Index**

	<b><u>Page</u></b>
1 Introduction and general information	4
1.1 Storage	5
1.2 Authorization for service technicians of our service partners	5
1.3 Description of the machine	5
1.4 Type label	5
2 Explanation of the safety symbols used	6
3 Use of the appliance for the purpose intended	7
4 General safety information	7
4.1 Operator's duty of care	7
4.2 Basic safety measures	8
5 Delivery, shipping, installation and assembly	10
5.1 Delivery	10
5.2 Transport, installation and assembly	10
5.3 Operating conditions	11
5.4 Requirements for the electrical connection	11
5.5 Requirements for the fresh water connection	12
5.6 Requirements for the waste water connection	12
5.7 Emergency-off	12
5.8 Chemicals for the operation of the appliance	13
6 Settings for initial commissioning by the service engineer	13
6.1 Commissioning	13
7 Cleaning with the Multi washer	14
7.1 Operating panel	14
7.2 Preparation for cleaning and rinsing	15
7.3 Automatic dosing	15
7.4 Operation during cleaning and rinsing cycle	15
8 Shutting down the Multi washer	16
9 Care and maintenance	16
9.1 Care, general	16
9.2 Refilling of detergent	16
9.3 Refilling with rinse aid	17
9.4 Cleaning	17
9.5 Care of stainless steel surfaces	17
9.6 De-scaling	18
10 Basic information on the appliance	18
10.1 General description of the cleaning machine	18
10.2 Noise level	19
10.3 Data reg. the electrical and hydraulic equipment	19
10.4 Dimensions, technical data, installation instructions	20
10.5 EC-/EU-Declaration of Conformity	20
11 Non-ionizing radiation	20
12 Tips for self-help in the case of faults	20
13 Staff training	21
14 Authorised user of this documentation	21
15 Settings / modifications / on-site adaptation	22
15.1 Using the keyboard for programming	22
15.2 Code entry	22
15.3 Service level	23
15.4 Parameter list	28
15.5 Assignment list View inputs / control outputs	32
15.6 Rinse program parameter update: 01.05.2008	33
16 Operating errors	34

16.1	Information reporting and troubleshooting	34
16.2	Error messages and troubleshooting	36
17	Maintenance	38
17.1	Basic safety measures during normal operation	38
17.2	Dosing units	39
17.3	Maintenance plan	40
18	Dismantling and disposal	42
18.1	Disposal of packaging materials	42
18.2	Dismantling and disposal of the old device	42
19	Documentation	42

# 1 Introduction and general information

Dear Customer,

We are delighted about the confidence you have shown in our products.

It is very important to us that you should obtain significant use from MEIKO products and that they should make your work easier.

If you follow the instructions in this document carefully, your Multi washer will always give you total satisfaction and will have a long service life.

The cleaning and disinfection machine has been assembled by us at the factory and has undergone a thorough inspection. This provides us with the certainty and you with the guarantee that you will receive a fully developed product.

**We would therefore ask you to read these operating instructions carefully before using the installation. Any further associated operating instructions for accessories and integrated third-party products must be strictly observed!**

These operating instructions inform users of this installation about the installation, its operating methods, its use, the safety instructions and the maintenance.

This information will help you to get to know the installation fully and to use it properly. It will also enable you to avoid repairs and the related loss of operational time.

In the event of any damage caused by non-observance of these operating instructions, any guarantee claims are invalid. This information will help you to use the installation properly.

MEIKO is constantly working on the further development of all its models.

We would therefore ask you to understand that because of this, we must reserve the right to make modifications at any time to any items covered by the contract in terms of their shape, fittings and technical characteristics.

No claims may therefore be based on the details, the images or the descriptions contained in these operating instructions.

Should you require any further information, or in case any particular problems not dealt with in great detail in the operating instructions should arise, you may contact the relevant MEIKO branch to obtain the information you require.

All MEIKO's obligations arise from the relevant purchase contract which also contains the entire and only valid guarantee provisions.

The operating instructions must exist in the local language for each EU country. If this is not the case, the machine must not be commissioned.

The original operating instructions in Germany, and all operating instructions in all languages for EU countries can be downloaded from the following address:  
<https://partnernet.meiko.de>

The complete technical documentation is issued to you free of charge.

Additional copies will be charged at cost.

These contractual guarantee rules shall be neither extended nor restricted as a result of any explanations given in the instructions.

The MEIKO Company very much hopes that you will enjoy our product and use it successfully.

## 1.1 Storage


Always store the operating instructions close to the installation!  
The operating instructions must always be kept within easy reach!

## 1.2 Authorization for service technicians of our service partners

MEIKO exclusively authorises authorised service partners for commissioning, inductions, repairs, maintenance, assembly and installation of the corresponding product groups within MEIKO devices.

## 1.3 Description of the machine

Please provide the following information on any query and/or when ordering spare parts:

Type:	_____
SN:	_____
	_____
<u>This information can be found on the plate.</u>	

## 1.4 Type label

The type label of the TopClean 60 is on the inside of the front panel.

## 2 Explanation of the safety symbols used

The following safety symbols will appear throughout these operating instructions. The purpose of these symbols is to draw the reader's attention to the text of the adjacent safety information.



This symbol warns that there is danger to human life and health.



This symbol warns that there is danger to the installation, to material or to the environment.



This symbol denotes information that helps you to understand the installation's operation.



Warning of dangerous electrical current!



Warning of possible hand injuries!



**No splashing water:** prohibits the use of a high pressure hose.



**Danger of explosion:** indicates a potential explosion hazard.



**Non-potable water:** The water is not for drinking. Health can be endangered by drinking.



**Danger of burning:** indicates possible hazard due to hot surfaces or media.



Eye protection must be used or protective glasses must be worn



Hand protection must be worn



Read the operating instructions

### 3 Use of the appliance for the purpose intended



The TopClean 60 Multiwasher disinfection machine is exclusively for its intended use, which is the cleaning of, for example surgical shoes, flower vases, trays etc.



The cleaning machine must be used only in accordance with regulations. Other uses are prohibited. The items to be cleaned must be suitable for cleaning in Multi washer.

This machine is intended solely for use in a commercial environment.

### 4 General safety information

#### 4.1 Operator's duty of care



The cleaning machine has been constructed based on a risk analysis and after careful selection of the applicable harmonized standards, as well as additional technical specifications. It therefore corresponds to the latest technology and is guaranteed to provide maximum safety.

This level of safety can only be achieved in practice, however, if all the necessary measures are taken. The operator of the installation has an obligation of care to ensure that these measures are scheduled, and also to check that they are correctly executed.

#### Measures to ensure the safe machine operation:



The operator must ensure in particular that ...

... the washing machine is only used in accordance with the regulations.

Should it be used in any other way, damage or danger may occur, for which we accept no liability (see the chapter on "Use for the Purpose Intended").



... in order to preserve the operational and safety guarantees, whenever required, only original parts supplied by the manufacturer are used.

the user will lose the right to any possible claims if the appliance is modified using any parts other than original parts.



... only appropriately qualified and authorized personnel use, maintain, and repair the installation.



... the relevant personnel is regularly trained in all questions relating to safety at work and environmental protection and, in particular, that they are familiar with the operating instructions as well as with the safety information provided in them.



... the installation is only operated in perfect, operationally efficient condition, all safeguards and cladding panels are installed, and, in particular, that the safety systems and switch elements are regularly checked for their operational efficiency.



... machines accessible only from behind may be operated only with rear panel cladding.



... the required personal protective equipment is made available to maintenance and repair personnel, and is worn by them.



... a functional test on all safety systems of the machine / installation is carried out during every regular maintenance.



... the operating instructions are always kept in legible, complete condition at the place where the installation is installed, and are always at hand.



.... any necessary initial tests to parts supplied by sub-suppliers must be carried out. More detailed information, if required, can be found in the relevant Instructions for Use.



after the installation, setting in operation and handing-over of cleaning machine to the customer/operator further modifications are not allowed (f. ex. electrical changes or changes in location). Any modifications to the cleaning machine without the written approval of MEIKO or undertaken by unauthorized persons will invalidate any right to claim and product liability.



... equipment for optimising energy consumption must not be used to reduce essential operating temperatures, as set out in DIN 10511, 10512 and 10522. If you, the client, install equipment for optimising energy consumption, any possible reduction in the quality of the wash and hygiene is your responsibility.

## 4.2 Basic safety measures



Danger can arise from the improper use of the machine or if it is used for purposes for which it was not intended.

Parts carrying electric current as well as moving or rotating parts can cause

- Dangers to the user's life and limb and
- Material damage



The machine may only be operated by adequately qualified staff who have been trained by the operating company and who have been trained about the Hazard and Safety Instructions.

Qualified staff, as defined by the Operating Instructions, are persons:

- who are over 14 years of age,
- who have read and who observe the safety instructions,
- who have read and who observe the operating Instructions (or the part applicable to the work to be carried out).



The machine operates with hot water. (Temperature of wash water = 58-60 °C, by cleaning and disinfection machine must be 74 °C. The machine operates with hot water.) Avoid all contact with the rinse water. Danger of scalding/burning! Please observe appropriate protective measures. Observe all the instructions posted on the machine.



### **Warning !**

When electrical equipment is in operation, it is inevitable that certain parts carry a dangerous current.

Before the machine's cover plates or an electrical device are opened it is imperative the entire machine is switched off completely via the on-site power disconnection device and that it is secured against reactivation by using appropriate measures.

Only specialist personnel may carry out repairs and rectification work on the electrical part of the machine. The Health and Safety Regulations must be observed.

The machine may be used again only after **all cladding panels** have been installed by the user of the machine.



The machine may not be sprayed with a water hose or high-pressure cleaner.



The machine must only be operated under the supervision of instructed staff.



The water in the wash-up area is non-potable and can't be used for food preparation!



If you are unsure about the operation of the machine, the machine must not be used.



Do not place any solvents or other easily flammable substances in the wash-up area, as this increases explosion hazard





Steel scrub pads are not to be used for the pre-scouring nor for cleaning the items to be cleaned. Do not clean any metal items in the machine which are not made of stainless steel.

The in-coming of metal parts (especially iron, tinplate, copper) must absolutely be avoided.

The appliance must not be used to transfer waste water from other sources into the drain (Warning: risk of corrosion and blockage).

Only use suitable products for cleaning the stainless steel surfaces, which do not attack the material, form any deposits, nor cause any discolorations.



Door and flaps **MUST** be closed.

Open the door very carefully during the programme cycle, as otherwise clean water could splash out.



The tank heating element may still be hot after the tank has been emptied. There is therefore the danger of burns when the machine is cleaned manually.



Only detergents and rinse-aids suitable for the use in industrial dishwashers may be used.

Corresponding information is submitted by the manufacturers of such products.

Detergents and rinse agents can be injurious to health.

The manufacturers hazard instructions on the original packaging and in the safety data sheets must be observed.



At the end of operation the machine is to be switched off completely with the on-site power disconnection device.

The accompanying Operating Instructions must be observed for accessory devices, e.g. water treatment installations.



**WE ACCEPT NO LIABILITY FOR DAMAGE OR INJURY  
ARISING FROM FAILURE TO OBSERVE AND ABIDE BY  
THESE SAFETY INSTRUCTIONS!!!**



#### **4.2.1 Working on electrical equipment**

Any repair work and repairs to the power supply on the installation's electrical equipment may only be carried out by a qualified electrician!

Check the electrical equipment regularly! Tighten any loose connections! Replace any damaged leads/cables immediately!

## 5 Delivery, shipping, installation and assembly

### 5.1 Delivery

Check that the delivery is complete immediately after receiving it by comparing it to MEIKO's contract confirmation and/or the delivery note.

If necessary, complain about any missing parts immediately to the shipping company and notify MEIKO.

Examine the appliance for possible transit damage.



Should you suspect any damage has occurred during shipping, you should inform:

- the shipping company,
- and MEIKO

in writing, and also send a photo of the damaged parts to MEIKO. In writing, and also send a photo of the damaged parts to MEIKO.



Damaged appliances must not be commissioned.

### 5.2 Transport, installation and assembly

In order to avoid damage or life-threatening injuries during shipping of the installation, the following points must be observed:



- The shipping operations may only be carried out by qualified persons who observe the safety instructions.
- Observe transport instructions on the packing.
- The appliance must be moved with great care.
- Unpack the machine.

In order to ensure safe shipping, the installation parts are placed on a special four-sided wooden frame.

Incoming goods should only arrive on these wooden frames. The packing is specifically designed to allow the appliances to be moved safely and securely using a pallet truck.

The enclosed technical sheet indicates the connection and consumption ratings of the appliance.



Small quantities of steam may escape from the door of the appliance. Furniture and equipment situated near the door must be protected.



An engineer from your local MEIKO Service Centre can install the appliance at the correct point and connect the tables - upon request.

The following must be observed during the installation of the Multi washer:

- The complete unit must be levelled in both directions using a water level.
- Compensate for an uneven floor by adjusting the feet.
- Table joints must be sealed with detergent-resistant sealing compound (e.g. silicone).

### 5.3 Operating conditions

It is taken for granted that the planning of the system, as well as installation, setting in operation and maintenance works are executed by sufficiently instructed staff and that these works are checked by responsible specialists. The indications on the name plate of the machine must correspond to the technical sheet and the local connection conditions.

Conditions to be provided by the customer:

- Frost free storage and installation area
- Electrical connection in accordance with the technical sheet
- Fresh water connection in accordance with the technical sheet
- Waste water connection in accordance with the technical sheet
- Anti-slip floor coverings should be provided around the Multi washer.

#### 5.3.1 Requirements for the installation area

- Ensure that the storage and installation area is permanently frost free.

The machine is only frost-resistant in the state it is delivered or when provided with special features (option: frost drainage). If the appliance is installed in an area where the surrounding temperatures are below freezing point, the water freezing inside can damage the internal water components such as pump, solenoid valve, boiler, etc.



### 5.4 Requirements for the electrical connection

Work on the electrical part of the machine may only be undertaken by specialist personnel.



The customer must guarantee the following points relating to the connection:

- The correct voltage and type of current must be available
- Safeguard the power supply cable according to regulations and provide it with a power disconnection device in the fixed electrical installation.
- The machine must be connected to the potential compensation system!
- If an unearthed neutral (N) is used with alternating current, the power disconnection device must have 4-poles (with alternating current 2-poles).
- For connection to three-phase current a 5-pole terminal strip (L1, L2, L3, N, PE) must be used.
- Electricity supply without neutral conductor (N): when connecting to three-phase current, use a 4-pole clamping strip (L1, L2, L3, PE).
- Conductor colors: live conductor L1 = black/1, L2 = brown/2, L3 = grey/3, neutral conductor N = blue/4, protective earthing conductor PE = green-yellow.

Current applicable standards and requirements of local utility companies are to be adhered to with regard to protective measures and connection of the potential compensation system.

The products are intended for permanent connection to the on-site power supply and have been tested for the market accordingly. Any other form of electrical connection is to be established by a licensed electrician.

Do not protect by fuses any additional consumers together with the Multi washer.

- All conductor fixing screws must be re-tightened before commissioning the appliance.



The wiring diagram is behind the front panel, resp. front cladding of the appliance. The enclosed wiring diagram must remain in the appliance.

#### **Note to customers**

Dishwashing machines, bedpan rinsing units and systems are intended for fixed, electrical power supply installation as well as connection to the locally available equipotential bonding and have been equipped with a corresponding connection option.

Operators may decide at their own discretion and responsibility to implement personal protection in locally available services in collaboration with a specialist electrician registered at the corresponding energy supplier using the following:

- AC/DC sensitive fault current protection switch with at max. 30mA EN 62423

or

- Automatic shutdown of the supply in the event of loss of protective earth conductor conductivity (EN 60204-1, Section 8.2.8.c)

## **5.5 Requirements for the fresh water connection**

Each appliance carries the DVGW test symbol and does not require an extra safety valve in the water feed.

- Fresh water connection must be made according to EN 1717 or according to local regulations.

**The machine is equipped with a water supply air gap (group A, model A in accordance with DIN EN 1717).**

- The requirements of the clean water supply in the accompanying GiO module operating and service manual are to be observed for machines equipped with the GiO module.



The minimum flow pressure of the clean water supply upstream of the solenoid valve must be 0.6 bar; if water-softening (AktiveClean) equipment is incorporated into the machine, it should be 1 bar; and for machines equipped with the GiO module it should be 1 bar.

The maximum pressure must not exceed 5 bar.

- If the flow pressure is below the minimum, increase the flow pressure with a booster pump; if the maximum pressure is exceeded, limit it with a pressure reducer.
- A water stop is integrated into the fresh water inlet of the machine. This, together with the leak water switch in the floor pan of the base, ensures that the fresh water supply will be cut off in the event of any leak
- Suitable protective measures must be taken to ensure that no iron particles can enter the appliance via the mains water supply. Similarly, precautions must be taken to prevent the entry of other metal particles, for example copper turnings. Corresponding instructions are contained in the installation drawing. Therefore suitable measures must be taken.
- A dirt trap must be fitted into the fresh water supply to protect the solenoid valve.

## **5.6 Requirements for the waste water connection**

- A waste water pump is integrated in the waste water line.
- The drain hose must be connected to the waste water pipe in the building.
- A grease trap may be needed, depending on the machine application.
- The requirements of the waste water supply in the accompanying GiO module operating and service manual are to be observed for machines equipped with the GiO module.

## **5.7 Emergency-off**

- Switch the machine off completely via the on-site power disconnection device.



## 5.8 Chemicals for the operation of the appliance

Only alkali detergents and acid rinse agents suitable for use in commercial dishwashers may be used. Corresponding information is submitted by the manufacturers of such products.

MEIKO recommends brand cleaning products from leading manufacturers. - cleaning and hygiene products are an excellent choice.

If unsuitable products are used, the life expectancy of the dosing units can be reduced considerably.

Observe the dosing instructions of the manufacturer.

Detergents and rinse-aids can present a health hazard if they are not correctly used.

The manufacturer's instructions on the original packaging and the technical data sheets must be observed.

"Items for rinsing are tribologically influenced in particular by chemicals and increased temperatures during the process, as well as mechanical stresses caused by handling and transporting."

If a de-scaling agent is used, please strictly observe the manufacturer's instructions reg. handling and safety. After having used such an agent, the product must be completely removed from the machine, as even small residues are sufficient to destroy plastic parts and packing materials.

### Chemical product settings

The correct settings for the quantity of detergent and rinse agent depend on the product used.

The relevant chemical supplier can install the correct setting.

## 6 Settings for initial commissioning by the service engineer

### 6.1 Commissioning

In order to avoid damage to the installation and the injury and death of persons when commissioning the installation, the following points must be observed without fail:

Any necessary initial tests to parts supplied by sub-suppliers must be carried out. More detailed information, if required, can be found in the relevant Instructions for Use.

- The installation may only be commissioned by suitably qualified persons observing the safety instructions.
- Before initial startup, check that any tools and parts not belonging to the installation have been removed.
- Check whether any escaping liquid is removed.
- Activate all the safety systems and door switches before commissioning.
- Check that all screw connections are tight.
- Please also read the chapter on "General safety instructions".
- Commissioning and instructions will be provided by technicians specially trained by Meiko. The operator may only use the installation after training has been provided.
- The "Commissioning certificate for GiO modules" is to be observed for machines equipped with the GiO module and the instructions adhered to accordingly.





## 7 Cleaning with the Multi washer

The appliance must not be used without a thorough knowledge of the "Operating Instructions". Incorrect operation could result in injuries to personnel or damage to the appliance.

### 7.1 Operating panel

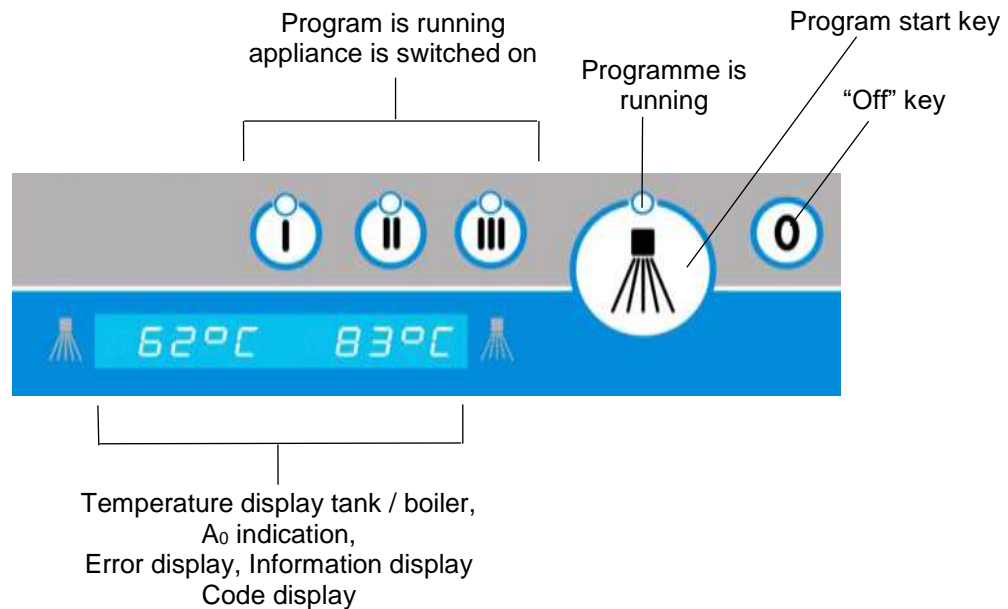


Illustration 1; Operating panel

Key / display	Meaning
	Short program – Clean program I
	Normal program – Clean program II
	Water change program – Clean program III
	Clean temperature
	Final rinse temperature
	Program start Tank drain Self-cleaning cycle
	Switch off appliance/ Cycle interruption

Table 1; Program key function / items to be cleaned

## 7.2 Preparation for cleaning and rinsing

The preparatory work described below must be carried out before each operation.



- Open the door.
- Insert the sieves. Insert the sieves.
- Close the door.



**ATTENTION!** Danger of crushing!

Close the door with both hands.



- Switch on the appliance by pressing one of the program pre-selection keys.

During the filling and heating phase, the light above the pre-selection key will flash. When the light remains constantly lit, the machine is ready for operation.

The time until the operation readiness is reached depends on the temperature of the supplied water and the installed boiler, resp. tank heating capacity.

In the case of cold water supply, the time taken is: about 18 minutes.

## 7.3 Automatic dosing

The required detergent and rinse aid is moved from the containers into the tank or boiler via electronically controlled dosing units. The dosing is effected automatically acc. to the requirements arising during the clean process.



If unsuitable products are used, the life of the dosing equipment will be significantly shortened.

We therefore recommend that detergents should have a pH value greater than 7 and that rinse agents should have a pH value between 7 and 2.

## 7.4 Operation during cleaning and rinsing cycle



The following fundamental principles must be observed when placing the items to be cleaned in the baskets:

- All hollow containers must always be loaded upside down. Otherwise the water will be trapped inside and they will not dry to a brilliant finish.
- Plates, trays and big plates should always stand at a **slight angle** in the basket. The inside faces pointing upwards.
- Do **not stack** the utensils in the wash basket directly on top of each other, as the clean water could not strike the items directly and unnecessarily long cleaning times would have to be selected. Short cleaning times with baskets which are not overloaded are much more economical.

### 7.4.1 Start the clean cycle

Program start key



- Pre-wash the dishware (major waste residues, paper, etc.) and place in the basket.
- Place the basket in the appliance, ensuring that it is correctly centred.
- Close the door.
- Press the program start key.

The appliance cleans and rinses automatically and switches off the clean program after completion. The program cycle is indicated by a light on the program start key.



The clean time can differ from the set program time if the boiler heating capacity or tank heating capacity is not sufficient for heating up the fresh water to the pre-set boiler temperature during the program time. In this case, the automatic clean time extension is activated. (For this, see chapter 10)

### 7.4.2 Remove the cleaned items

- When the light goes out, open the door and remove the basket.

"Off" key



Program start key



## 8 Shutting down the Multi washer

- Press the "0" key (OFF key). The machine is switched off when all the lights are out.
- Press the program start key to drain the tank.
- The tank interior is sprayed with clean hot water after the tank water has been drained. The door must remain closed. The waste water pump switches off automatically.

## 9 Care and maintenance

### 9.1 Care, general

The appliance has been designed to minimise the need for cleaning, care and maintenance.



However, for a reliable, safe and permanent function of the appliance and in the interest of hygiene and cleanliness a correct care and maintenance is necessary. To facilitate this procedure, a maintenance contract can be concluded with the manufacturer or the manufacturer's agent.



Works/repairs which were not correctly executed and the use of unauthorised parts by unqualified personnel endanger both operators and the appliance, and will invalidate the warranty.

### 9.2 Refilling of detergent

There are two different types of detergent containers:

#### Incorporated container

The storage container is located behind the front panel in the lower part of the appliance. The front panel can be tilted towards the front after a slight lifting.

- Re-fill the container marked "detergent" if necessary.

#### External container

The container is located next to the appliance.

- Check the filling level of the container and if necessary, replace it by a full one

Only non-foaming alkali detergents (pH > 7) suitable for commercial cleaning machine may be used.



Detergent dosing units must be checked to see if they are functioning properly if there is reason to believe that they are malfunctioning. Carry out a visual inspection!



### 9.3 Refilling with rinse aid

There are two different types of rinse aid containers:

#### Incorporated container

The storage container is located behind The front panel can be tilted towards the front after a slight lifting. The front panel in the lower part of the cleaning machine.

- Re-fill the container marked "detergent" if necessary.

#### External container

The container is located next to the appliance.

- Check the level and, if necessary, replace the container by a full one

Only non-foaming alkali detergents (pH > 7) suitable for commercial cleaning machine may be used.

Rinse dosing units must be checked to see if they are functioning properly if there is reason to believe that they are malfunctioning. Carry out a visual inspection!



### 9.4 Cleaning

After the tank has been drained, proceed as follows:

- Do not use a foaming detergent for the machine by hand for pre-cleaning close to the Multi washer. Foam can cause malfunctions in the Multi washer and a poor clean.
- Residues sticking to the tank, tank heating element and sieves must be removed with a brush.
- Dismantle the wash arms and rinse them under running water.
- Wash nozzles must be cleaned daily.
- The cleanliness of final rinse nozzles must be checked weekly and if necessary clean under running water.

#### 9.4.1 Safety instructions for cleaning

The tank heating element may still be hot after the tank has been emptied. There is therefore the danger of burns when the machine is cleaned manually.



The machine, switch cabinet and other electrical components must NOT be sprayed with a hose or a high pressure cleaner.

### 9.5 Care of stainless steel surfaces

We recommend cleaning the stainless steel surfaces only when needed with cleaner and care products suitable for stainless steel.

Lightly soiled parts can be wiped with a (possibly damp) cloth or sponge.

Be sure to wipe dry after cleaning to avoid traces of scale. Use demineralised water if possible.

Do not use aggressive cleaning or scouring agents.

The care products must not attack the stainless steel, form deposits, or cause discoloration.

Never use cleaning agents that contain hydrochloric acid or bleaches based on chlorine.

Never use cleaning equipment that you have used previously by non-stainless steel to avoid external corrosion.

Aggressive external influences due to cleaning and care products that evaporate in the vicinity of the dish-washing machine, or caused by direct application, can lead to machine damage and put the material at risk (e.g., aggressive tile cleaners).

#### Caution!

Respect the safety rules of the manufacturers on the original packing as well as on the safety data sheets.

## 9.6 De-scaling

If the appliance was operated with hard water, the boiler and wash tank could have lime scale deposits. De-scaling of the tank interior, boiler housing, tank heating, boiler heating and clean and final rinse system then becomes necessary.

For de-scaling the appliance use only products suitable for industrial washing machines. Please observe the instructions of the manufacturers of such products.



After de-scaling the appliance:

- Remove the de-scaling agent completely from the appliance. 1 or 2 rinse cycles with fresh water are necessary to achieve this.



Even small residues of de-scaling agents can be sufficient to destroy plastic parts and sealing materials! If the appliance is heavily scaled, you should ask a service engineer from the agency responsible to de-scale the boiler.

## 10 Basic information on the appliance

Each Multi washer is manufactured acc. to the latest state of the art technology. Operation is safe.



Dangers could arise from this model, if it is not correctly operated by unsuitable operating staff or if it is not used acc. to its purpose.



### Liability

We accept no responsibility for damage of the appliance and other objects caused by operating faults, resp. non-observance of the operating instructions. Any modifications to the appliance - especially technical modifications inside - undertaken by unauthorised persons without the written permission of the manufacturer will invalidate the warranty.

### 10.1 General description of the cleaning machine

#### 10.1.1 Execution

Square basket appliance with stationary basket

#### 10.1.2 Clean principle

The appliance has one clean and one final rinse cycle.

The temperature regulator maintains the cleaning temperature of approx. 74°C. A centrifugal pump circulates the water out of the cleaning tank into the wash nozzles. The water jets reach the items to be cleaned out of differing directions. Therefore an even cleaning result can be guaranteed.

The cleaning cycle is followed by the fresh water final rinse. The items are rinsed via a separate nozzle system with hot fresh water at approx. 83° C. Thus heating up the items for the following drying process. At the same time the final rinse water serves for the regeneration of the clean water, the level of soil of the cleaning water thus being reduced.

#### 10.1.3 A<sub>0</sub> control

The standard factory setting is A<sub>0</sub> = 60.

The tank temperature for cleaning is 74 °C. Tank heating is active while cleaning. After every second as of 65° C in the clean tank the measured tank temperature is allocated one factor (the higher the temperature, the higher the factor). These factors are continually added until the desired hygiene value, e.g. A<sub>0</sub> 60, has been reached. The dripping phase and final rinse begin when the clean time set in the programme is reached or exceeded.

The display indicates the A<sub>0</sub> value.



This method makes it possible to achieve disinfection levels higher than the standard (e.g. in hospitals).



The tank temperature falls when the programme begins, depending on the washware. The time needed to reach the set disinfection parameters could exceed the set programme time.



High cleaning temperatures and long retention times in the clean tank can lead to glass corrosion and premature peeling of the decor.

#### 10.1.4 Water change program

The program pre-selection keys can be allocated to a water change programme. In the standard setting this applies only to button III.

All of the tank water is pumped out when the clean is complete. Fresh water rinsing follows, and this water is pumped out as well. Then the programme ends and the programme start button light goes out.

Now the following options are possible:

1. Press the "0" key (OFF key), and the machine stays off.
2. Open the door, remove the basket, close the door, and then the machine is ready to use. (fill tank, heat)
3. Switch to programme 1 or 2, and then the machine is ready to use. (fill tank, heat)
4. Press the start key, and then the machine is ready to use (fill tank, heat) and then the clean program starts directly afterward.

#### 10.1.5 Detergent dosage

The detergent dosing unit is designed for the automatic addition of liquid alkaline detergent to the clean water.

The detergent is transported out of the container into the clean tank by means of a hose line. The dosing unit is self-priming. The dosing is effected during each filling cycle and at the beginning of each programme cycle using timer control.



Normally, a dosing of approx. 2 ml of detergent per liter of tank water is the correct concentration. This can be increased/reduced acc. to the water quality, items to be cleaned and degree of soiling to 5 ml/l or to 1 ml/l.

#### 10.1.6 Rinse aid dosing

The rinse aid dosing unit is designed to automatically add liquid final rinse aid into the fresh water. The rinse aid is transported out of the container into the fresh water supply line by means of a hose line. The dosing unit is self-priming. The dosing takes place during each filling cycle.



The correct dosing results in a smooth, even water film.

In case of overdosing, there are bubble and stripe formations - reduce dosing.

In case of under-dosing, water drops remain on the cleaned items - increase dosing.

### 10.2 Noise level

Work place noise level L<sub>pA</sub> ≤ 70 dB

### 10.3 Data reg. the electrical and hydraulic equipment

See attached technical sheet

## 10.4 Dimensions, technical data, installation instructions

See enclosed documentation

## 10.5 EC-/EU-Declaration of Conformity

See separate EC-/EU-Declaration of Conformity

## 11 Non-ionizing radiation

Non-ionizing radiation is not produced intentionally but unfortunately comes about due to electrical operating equipment (e.g. electrical motors, high-voltage cables and magnetic coils). In addition the machine has no strong permanent magnet. There is a high possibility of eliminating the influence of active implants (e.g. pacers, defibrillators) by maintaining a safety distance of 30 cm (distance of the field source to the implant).

## 12 Tips for self-help in the case of faults

Fault:	Cause
Machine does not fill.	<ul style="list-style-type: none"><li>• No water available</li><li>• Dirt trap blocked</li><li>• Level switch defective</li><li>• Solenoid valve defective</li><li>• Door safeguard defective</li></ul>
Rinse water does not spray!	<ul style="list-style-type: none"><li>• No water available</li><li>• Dirt trap blocked</li><li>• Solenoid valve defective</li><li>• Booster pump has failed</li><li>• Fresh water rinse system furred</li></ul>
Stripes and smears on the utensils!	<ul style="list-style-type: none"><li>• Rinse water mineral content too high</li><li>• Water pre-treatment defective or not carried out</li><li>• Different water type depending on the waterworks</li><li>• Unsuitable rinse aid products or wrong dosage quantity</li></ul>
Formation of a significant amount of foam in the clean tank!	<ul style="list-style-type: none"><li>• Detergent for Multi washer by hand enters the cleaning tank because of pre-cleaning the utensils</li><li>• Daily cleaning is carried out with foaming cleansing agents which afterwards enter the machine.</li><li>• Improve pre-wash, as too much food residue is entering the tank. Use a water change program as an alternative.</li><li>• Rinse water quantity too low</li><li>• Detergent or rinse aid product not suitable</li><li>• Temperatures too low &lt; 40°C</li></ul>

## 13 Staff training

Only trained and instructed personnel are allowed to work on the Multi washer. Staff responsibilities for the installation's operation, maintenance and repair must be clearly defined.

Any personnel undergoing training are only allowed to work on the Multi washer installation under the supervision of an experienced person.

<b>persons</b> <b>Activity</b>	<b>Trained operating personnel</b>	<b>Trained in-house technician</b>	<b>Trained in-house technician or installation engineer</b>
Installation and assembly			◆
Commissioning			◆
Operation, use	◆	◆	◆
Cleaning	◆	◆	◆
Checking safety devices	◆	◆	◆
Fault finding		◆	◆
Troubleshooting, mechanical		◆	◆
Troubleshooting, electrical			◆
Maintenance			◆
Repairs		◆	◆

Training should be recorded in writing.


## 14 Authorised user of this documentation



The works described in this booklet (chapter 15 - 19) may only be carried out by specialists of the manufacturer, the responsible agency or an authorized dealer.

## 15 Settings / modifications / on-site adaptation

### 15.1 Using the keyboard for programming

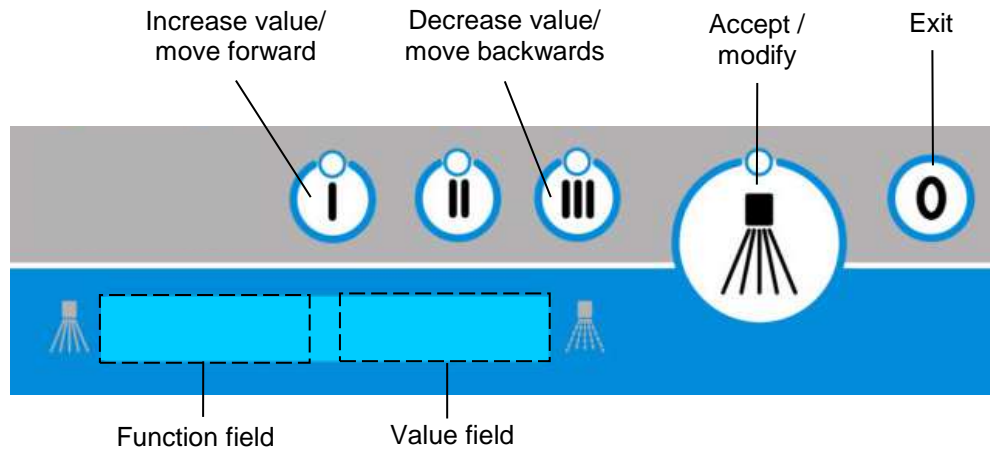

Access codes for various user-levels have been defined. Once the complete code has been entered, the entered code is compared with the internal code table. Depending on the code entered, the corresponding user level will be accessed. 2 access codes are available for each user level; the first is for restricted access, i.e. no modification of parameters is possible (viewing mode), and the second gives access to the entire range of functions (viewing and modification).

In the short programming instructions accompanying every machine in the series, this is described in condensed form.

For control programming, the power supply must be available but the machine must be completely switched off (no LED must be illuminated).

#### Code – input:

View service data:	CODE 10000
Modify service data:	CODE 10001
View configuration data:	CODE 20000
View dosing technology:	CODE 40000
Modify dosing technology:	CODE 40044

The code numbers for the further levels can be found in the Service Manual.

### 15.2 Code entry

To get into the code entry mode, you should keep the key “0” pressed (for around 3 seconds) until you see



on the display unit..

By pressing the key “0” once again you can leave the programming area at any time.

The digit to be modified will flash.

Press the “I” key to increase the value/code indicated on the display unit, or press the “III” key to decrease it, or press the “accept” key to save it. The next value will then flash and will be the only one visible.



If your entry is incorrect you will exit the code entry procedure, and the information code 122 will be displayed.



If you enter all the digits correctly you will arrive at the chosen level, either service, configuration or machine data.

### 15.3 Service level

The list of service parameters can be found on this level (parameter numbers 1xx). Here you can view these or modify them, or you can also call up the ventilation of the rinse and clean hoses.

On the service level, you will first see the display below:



This corresponds to the viewing/modifying parameters (see 15.3.1)



This corresponds to rinse aid inlet ventilation (see 15.3.2)



This corresponds to detergent inlet ventilation (see 15.3.3)

Press the "I" key to move forwards or the "III" key to move backwards or the "accept" key to make a selection. You are now at the current level.

You can leave this level by pressing the "0" key.

### 15.3.1 View/modify parameters

Indication



this display by pressing the “accept” key.

Now, the first parameter will be displayed with a value.



Press the “I” key to go forwards and the “III” key to go backwards, until the parameter you require is displayed.

Confirm the parameter to be modified by pressing the “accept” key, the value will flash. Press the “I” key to increase the value, the “III” key to decrease the value, and the “accept” to save the value. You can leave this level by pressing the “0” key.

See 15.4 for list of parameters

### 15.3.2 Ventilating the rinse aid inlet



this display by pressing the “accept” key.

Now the dosage pump will be activated and the remaining running time will be indicated.



You can leave this level by pressing the “0” key. The ventilation will be interrupted off.

### 15.3.3 Ventilating the detergent inlet



this display by pressing the “accept” key.

Now the dosage pump will be activated and the remaining running time will be indicated.



You can leave this level by pressing the “0” key. The ventilation will be interrupted off. Should the ventilation process be insufficient, repeat the process.



The function "vent detergent pipe" is not existing with machines including detergent dosing system type ADT ('Advanced Dosing Technology with depression dosing). The detergent dosing pipe is vented automatically when the first cycle is running, after filling of the detergent reserve container.



### 15.3.4 Configuration level

You can find the list of configuration parameters on this level (parameter numbers 2xx). + Here you can view these and modify them. You can also call up the status of the inputs and outputs, or set the outputs for testing.

On the configuration level, you will first see the display below:



This corresponds to the viewing/modifying parameters (see 15.3.5)



This corresponds to viewing the status of inputs. (see 15.3.6)



This corresponds to viewing and setting the status of outputs (see 15.3.7).

Press the “I” key to move forwards or the “III” key to move backwards or the “accept” key to make a selection. You are now at the current level.

You can leave this level by pressing the “0” key.

### 15.3.5 Viewing / modifying parameters: (depending on the code entered)

Confirm



this display by pressing the “accept” key.

Now, the first parameter will be displayed with a value.

Press the “I” key to move forwards or press the “III” key to move backwards, until the parameter you require is displayed.

Confirm the parameter to be modified by pressing the “accept” key, the value will flash. + Press the “I” key to increase the value, the “III” key to decrease the value, and the “accept” to save the value.

You can leave this level by pressing the “0” key.

See 15.4 for list of parameters

### 15.3.6 Viewing input status:

Confirm



this display by pressing the “accept” key.

by pressing the “accept” key.



Press the “I” key to move forwards and the “III” key to move backwards, until you reach the input you require.

Display: input set



Display: input not set



You can leave this level by pressing the “0” key.

Assignment details for the inputs are given on the assignment list for each machine (see 15.5).

### 15.3.7 Viewing / modifying output status (according to code entered)

Confirm



this display by pressing the “accept” key.

**Viewing:**

Now, the first output will be shown, with status.



Press the “I” key to move forwards or press the “III” key to move backwards, until the output you require is displayed.

**Modifying:**

Press the “accept” key to confirm the modification of the output, the value will flash.  
Press the “I” key to modify the value and press the “accept” key to save it.  
The output is now set.



You can leave this level by pressing the “0” key.

Assignment details for the outputs are given on the assignment list for each machine (see 15.5).

**15.3.8 Viewing / modifying dosing technology level**

By entering code 40000 (read only) or 40044 (read / enter), the user can access the new 4th parameter level summarizing all the dosing technology parameters:

P104, P105, P218, P219, P224, P225, P321, P322, P326, P327.

See 15.4 for list of parameters

## 15.4 Parameter list

Par. No.	Configuration options	Use as	value range	Unit	Factory setting	Note
101	Wash program Key 1	Indication	1 .. 50	-	23	Allocate the wash program to the key 1 Assignment adjustable
102	Wash program Key 2	Indication	1 .. 50	-	27	Allocate the wash program to the key 2 Assignment adjustable
103	Wash program Key 3	Indication	1 .. 50	-	39	Allocate the wash program to the key 3 Assignment adjustable
104	Rinse agent Dosing quantity	Indication	0.10 .. 1.00	ml/Liter water	0.2	Value can be read from the rinse aid container label (dependant on water quality)
105	Detergent Dosing quantity	Indication	0.1... 20.0	ml/Liter water	2.0	Value can be read from the detergent container label (dependant on water quality)
106	Hardness degree	Parameters	0 .. 50	[°dH]	0	The quantity of soft water available between two regenerations depends on the hardness of the water.
107	Beep ON/OFF	Parameters	0/1	-	1	Switch on/off acoustic ready message
108	Mode "Clear" display	Parameters	0/1	-		"Clear" display 0: via INFO 420, 520 1: display of special characters
109	Partial / full desalination available?	Parameters	0,1,2	-		Partial / full desalination available? 0: NO 1: partial demineralisation (TE) 2: full desalination (VE)
110	Hardness litres per cartridge type	Parameters	0 .. 250	1000 L		When the cartridge's capacity is reached (hardness litres/degree of hardness), "Replace Cartridge" will be displayed (INFO 725) (only in the case of TE)
111	Total Operation time Display	Display	5 figures	h		Operation time query only
112	Total number of wash cycles	Display	5 figures	-		Wash cycles/loads, query only
113	Total number of wash cycles since last reset	Display	5 figures	-		Wash cycles/loads, re-setting possible
114	Serial number	Indication	8 figures	-		Option for calling up works parameters
115	Condition Remaining cartridge capacity	Indication	0 .. 100	%		Only for partial / full desalination TE: indication in % VE: 100 = OK; 0 = replace
119	IR-communication	Parameters	0/1	-	1	It is possible to shut off communication via IR interfaces. (0)

Par. No.	Configuration options	Use as	value range	Unit	Factory setting	Note
120	Total number of wash cycles Indication	Parameters	0/1	-	0	Definition: Activation detergent pump: <b>ATTENTION!</b> All changes to service parameters will be reversed. Power supply reset must be carried out within 5 minutes, otherwise factory settings will not be loaded. Without power supply reset, the information 123 will be displayed.
201	Machine type	Parameters	1 - 9	-	6	1: FV 40.2 / FV 60.2 / FV28 GiO-M 2: FV 130.2 / FV 250.2 / DV 270.2 3: DV 80.2 / DV 200.2 4: DV 120.2 / DV 125.2 / DV 200.2PW 5: FV 70.2D / FV 40.2TL / TopClean60 6: FV 130.2 TL / FV 250.2 TL / DV 270.2 TL 7: DV 80.2 TL / DV 200.2 TL 8: DV 120.2 TL / DV 125.2 TL / DV 200.2 TL PW <b>Attention!</b> Only assignment list and machine sequences change – no parameters
202	Must be-tank temperature	Indication	10 ... 80 (50 .. 176)	°C/°F	74	Standard for all the rinse programs on one appliance! Output dependent on definition.
203	Pre-rinse time	Parameters	0 ... 8	sec.	0	See pre-rinse process step
204	Rinse time	Parameters	4 ... 30	sec.	5	5: FV 40.2/TopClean 60 6: FV 60.2 Energizing duration for the pressure increasing pump (running time limited by P306!!)
205	Indicator lamp	Parameters	0 .. 8	-	1	Definition of the information which is to be switched via the potential-free contact 0 – no information 1 – filling/Heating, ready for washing/washing or pumping out 2 – filling/heating, ready for washing/washing 3 - filling / Heating 4 - ready for washing 5 - washing 6 - draining 7 - error 8 – not status machine OFF and Draining 9 - reserve 10- not status Machine OFF
211	Fine adjustment Rinse time	Parameter	0,0..0.9	sec.	0.7	0,7: FV 40.2 0,5: FV 60.2 Figures after the decimal point in P204

Par. No.	Configuration options	Use as	value range	Unit	Factory setting	Note
218	Shortage of rinse aid	Parameters	0/1		0	Monitoring Indication
219	Shortage of detergent	Parameters	0/1		0	Monitoring Indication
224	Activation mode rinse agent pump.	Parameters	0 .. 4	-	1	Definition: Energizing rinse aid pump: 0 – rinse pump = 0; no signal 1 – rinse aid pump; energizing according to calculated running time 2 – rinse aid pump = pressure increasing pump; energizing as pressure increasing pump 3 – rinse aid pump = wash pump; energizing as wash pump 4 - free
225	Energizing mode detergent pump	Parameters	0 .. 4		4	Definition: Energizing detergent pump: 0 – detergent pump; no signal 1 – detergent pump; energizing according to calculated running time 2 – detergent pump = pressure increasing pump; energize as pressure increasing pump 3 – detergent pump = wash pump; energize as wash pump 4 – option – detergent pump using negative pressure dosing
228	Water softener incorporated?	Parameters	0/1	-	0	In case of incorporated water softener set to 1
240	Load factory settings for configuration data	Parameters	0/1	-	0	Effective only upon power supply reset ON/OFF <b>ATTENTION!</b> All changes to service parameters will be reversed Power supply reset must be carried out within 5 minutes, otherwise factory settings will not be loaded. Without power supply reset, the information 123 will be displayed.
241	A <sub>0</sub> -value	Parameters	0 ...60	-	60	Only with disinfection machine no. 5 - 9 in parameter 201
321	Rinse agent pump output	Parameter	0.1 ... 10	l/h	1.3	Rinse agent pump. Output definition.
322	Detergent pump output	Parameter	0.1 ... 20	l/h	8.5	Detergent pump Output definition.
326	Pipe vent time rinse agent	Parameters	0 ... 255	sec.	200	Activate rinse agent pump temporarily to remove air from pipe.
327	Pipe vent time detergent	Parameters	0 ... 100	sec.	0	Activate detergent pump temporarily to remove air from pipe.

Par. No.	Configuration options	Use as	value range	Unit	Factory setting	Note
347	Desinfection temperature	Parameters	10 ...80	°C/°F	0	Only with disinfection machine no. 5 - 9 in parameter 201
348	Desinfection temperature	Parameters	0 ...900	sec.	0	Only with disinfection machine no. 5 - 9 in parameter 201

## 15.5 Assignment list View inputs / control outputs

Indication			Input / output / other	Conditions
Left	Right			
In	1	0/1	Door closed	none
In	2	0/1	Boiler level	none
In	3	0/1	Leak water switch floor	none
In	4	0/1	not occupied	none
In	5	0/1	not occupied	none
In	6	0/1	not occupied	none
In	7	0/1	Hall-sensor ADT (vacuum dosing)	none
In	8	0/1	not occupied	none
In	9	0/1	not occupied	none
In	10	0/1	not occupied	none
In	12	0/1	not occupied	none
In	13	0/1	Threshold tank level 1	none
In	14	0/1	Threshold tank level 2	none
In	15	0/1	Threshold tank level 3	none
In	16	0/1	tank level 4 (option)	none
In	17	0 .. 255	Without function	none
In	18	0 .. 255	Without function	none
In	19	xxx	Boiler temperature in °C or °F	none
In	20	xxx	Tank temperature in °C or °F	none
In	21	xxx	Tank level (1 mm unit)	none
In	22	0 .. 255	Without function	none
In7	9	0/1	Level switch in water softener inlet chamber	none
In7	10	0/1	“No salt“ switch	none
In7	18	0/1	not occupied	none
In7	19	0/1	not occupied	none
Ou	1	0/1	Wash pump	No leak water
Ou	2	0/1	Booster pump	No leak water
Ou	3	0/1	Drain pump	No leak water
Ou	4	0/1	Rinse aid – dosage pump	No leak water
Ou	5	0/1	Detergent – dosage pump / valve from HDT	No leak water
Ou	6	0/1	Operation indicator	No leak water
Ou	7	0/1	Filling valve	No leak water
Ou	8	0/1	SASm soft starter system	No leak water
Ou	9	0/1	Boiler heating	No leak water
Ou	10	0/1	Tank heating	No leak water
Ou	11	0/1	not occupied	none
Ou	12	0/1	not occupied	none
Ou7	4	0/1	Booster pump water softener	No leak water
Ou7	5	0/1	not occupied	none
Ou7	6	0/1	not occupied	none
Ou7	8	0/1	Solenoid valve Y2 EW	No leak water
Ou7	9	0/1	Solenoid valve Y3 EW	No leak water
Ou7	10	0/1	Solenoid valve Y4 EW	No leak water
Ou7	11	0/1	Solenoid valve Y5 EW	No leak water
Ou7	12	0/1	not occupied	none

Leak water switch condition: Leak water switch must not have operated.



## 15.6 Rinse program parameter update: 01.05.2008

Program no.:	Boiler temperature target value	Wash time target value	
		Washing	Total
1	83	71	90
2	83	101	120
3	83	221	240
4	65	71	90
5	83	141	160
6	83	161	180
7	83	191	210
8	83	341	360
9	65	101	120
10	65	141	160
11	65	161	180
12	65	191	210
13	65	221	240
14	65	341	360
15	85	71	90
16	85	101	120
17	85	141	160
18	85	221	240
19	85	341	360
20	83	251	270
21	83	281	300
22	83	311	330
23	83	341	360
24	83	371	390
25	83	401	420
26	83	431	450
27	83	461	480
28	83	491	510
29	83	521	540
30	83	71	90 (WW)
31	83	101	120 (WW)
32	83	131	150 (WW)
33	83	161	180 (WW)
34	83	191	210 (WW)
35	83	221	240 (WW)
36	83	251	270 (WW)
37	83	281	300 (WW)
38	83	311	330 (WW)
39	83	341	360 (WW)
40	83	371	390 (WW)

WW = Water change program



The dosage times will be adapted to the rinse time, so that the correct concentration remains if the rinse time is modified.

## 16 Operating errors

Despite being expertly designed, the machine may develop minor faults which are usually easy to eliminate. This section explains a number of possible problems and how you can deal with them yourself.



Before carrying out work on the open appliance, it **MUST** be disconnected from the power supply. The machine is to be switched off completely with the on-site power disconnection device here.

Should any of the operational faults described arise repeatedly, their cause must be established in each case.



Faults not described here can in general only be eliminated by a technician or electrician. Please contact the agency responsible or the authorised dealer.

### 16.1 Information reporting and troubleshooting



Figure 2: Information display

Information displays can be cleared by pressing the confirmation key.

Provided that the machine function is restored, the next program sequence will begin. The information display can also be deleted by pressing the "Off" key.

Information indicator (extract)

INFO-no.	Description	Possible cause
120	Emergency program active Restricted washing possible.	No boiler / tank heating No fresh water supply Check system
121	Door not closed	Check connection S1 Change microswitch Check microswitch adjustment Replacing a defective I/O circuit board
122	Incorrect password / no authorization	Enter code once again
123	Factory setting parameter list	Switch power supply ON/OFF within 5 minutes and set parameters back to factory settings. This will be rejected and parameters will be retained. Information 123 will disappear
126	Maintenance required	The set operating hours (P122) or batch number (P123) has been reached. Inform the service department and perform maintenance. Reset the maintenance counter (P124)
420	Shortage of rinse agent	If the machine is ready for operation, a shortage of rinse agent will be signalled (only if there is a built-in warning system).

INFO-no.	Description	Possible cause
520	Shortage of detergent	If the machine is ready for operation, a shortage of detergent will be signalled (only if there is a built-in warning system).
521	Shortage of detergent with dosing system ADT	Impulses of the flow meter are recognized, although the detergent dosing is not approached. Valve of the dosing unit is not closing.
522	Error in the dosing system ADT	Impulses of the flow meter are recognized, although the detergent dosing is not approached. Valve of the dosing unit is not closing.
720	Regeneration in progress	Regeneration program has started and is in progress. (It is possible to pause the program, but not cancel it.)
721	Regeneration is not possible as no salt available	Regeneration is necessary, but no brine available
722	Refill with salt	No brine available: Check salt supply No water in the salt solution reservoir. Float switch S5 not working.

Table 2: Information displays

## 16.2 Error messages and troubleshooting



Figure 3: Error messages

Error messages will disappear automatically when the fault has been rectified.

Error messages (extract)

ERR.-No.	Description	Possible cause
001	EEPROM plug-in fault.	EEPROM not available / incorrectly plugged in / defective Empty or incorrect EEPROM Replace EEPROM with correct parameter set
111	Floor pan leakage	Leak inside the machine Pump sump / motor /etc. Defective leak water switch Repair fault, remove water
117	Door not locked	The pin of the lifting magnet is not correctly in the locking device The magnetic coil of the lifting magnet is damaged Signal of door locking is not correct
201	Level not reached during 1st filling	Fresh water inlet insufficient (water faucet closed) AquaStop hose kinked Inlet filter soiled AquaStop defective Boiler switch defective
202	Level not reached early enough during filling	See 201
203	No change detected by the level switch during emptying	Boost pump defective Plug connector loosened Start capacitor defective Plug connector loosened Plug connector loosened Boiler level switch defective No boost pump signal to - from input/output circuit boards Check boost pump / S2 using manual control
204	Following the end of rinse time, still no change detected at the level switch	See 203
205	Temperature increase not reached	Boiler heating defective / thermal fuse radiator Temperature sensor defective, incorrect installation position Boiler contactor defective, performance switch loosened No signal from ON/OFF circuit boards

ERR.-No.	Description	Possible cause
206	Wash time increase	Boiler not ready for rinsing early enough (boiler level/ boiler temperature) Boiler heating defective / thermal fuse radiator Temperature sensor defective Boiler contactor defective, performance switch loosened No signal from ON/OFF circuit boards
210	Temperature sensor short circuit	Check sensor cable (plug contacts) Replace sensor Install sensor correctly
211	Temperature sensor interruption	See 210
212	Actual boiler temperature too high	Contactor sticking Incorrect sensor / defective sensor Check sensor / cable (contact plug MIKE II XA5)
301	Number of circulatory pumping cycles exceeded. Tank level analysis disrupted	DSP yield too low Rinse jets soiled Air trap soiled Booster pump rotor defective Condensate in level pipe Hose kinked / loose / not watertight
302	When pumping out during rinse program, level does not fall below level 1.	Fresh water supply insufficient (water faucet closed) Drain pump yield too low Drain pump soiled / defective Rotor loose Drain pump plug connector loose Start capacitor defective Tank level analysis disrupted AquaStop not closing completely No signal from ON/OFF circuit boards
303	Level does not fall below level 3 after time (drain pump ON)	See 302
304	Temperature increase not reached	Tank heating defective / thermal fuse Radiator Temperature sensor defective, incorrect installation position Tank protection defective, performance switch loose
305	Boiler content quantity insufficient for rinsing. Level 2 not reached	See 301 Level switch defective Plug connector loosened
306	Tank level analysis disrupted Tank level analysis disrupted	Ventilation valve soiled Check tank level Level sensor air catch / check hose
307	Tank level sensor defective	Connection plug loosened Sensor defective Replace input/output circuit boards
310	See 210	See 210
311	See 211	See 211
312	See 212	See 212
502	Shortage of detergent	Shortage of detergent is reported for a machine that is ready for operation

ERR.-No.	Description	Possible cause
701	Intermediate water softener reservoir does not fill	Water supply turned off Water softener level switch is not operating or is defective AquaStop valve Y1 defective
702	Intermediate water softener reservoir does not empty	Booster pump in AktivClean actually overheated or defective Valve in AktivClean not operating Water softener level switch is not operating or is defective

Table 3: Error messages

Should information or fault numbers not shown in the tables be indicated, or should the suggested measure not lead to the elimination of the fault, please notify a customer service technician.

## 17 Maintenance

Regular maintenance is a prerequisite for the long-term reliable and safe operation of a warewashing machine. Maintenance which is neglected or improperly carried out increases the residual risk of unforeseen damage to property and persons, for which no liability will then be assumed.

Maintenance work should only be conducted if the machine has been switched off completely via the on-site power disconnection device.

### Existing safety systems may not be removed!



A functional test on all safety systems of the machine / installation is carried out during every regular maintenance

We recommend that you take out a maintenance contract with our manufacturer's agent in order to ensure a long service life.

### 17.1 Basic safety measures during normal operation

Observe the maintenance periods prescribed in the operating instructions!

Observe the maintenance instructions given in these operating instructions for individual components!



#### Danger of injury from entering a danger zone

Unauthorised persons might be in or enter the danger zone during transport, assembly, commissioning, maintenance and repair work. This can lead to injuries.

- Only permit qualified persons to perform work at the machine.
- Remove unauthorised persons from the danger zone.
- Cordon off danger zone and signpost it for third parties.
- Never remove or disable safety devices on the machine.
- Always wear cut-resistant protective gloves when removing housing parts and when working inside the machine!



Before implementing any maintenance or repair work the machine must be switched off completely via the on-site power disconnection device and secured against reactivation by using appropriate measures (e.g. via a padlock whose key is in the possession of the person conducting the maintenance or repair work)!

Failure to observe these precautions can result in severe physical injury or damage to property.



Before carrying out any maintenance and repair work, ensure that all the parts of the machine that may be touched have cooled down to room temperature!

Carefully dispose of any cleaning products that could harm the environment!

### **17.1.1 Before starting operations following maintenance or repair work**



Before starting operations following maintenance or repair work, all initial tests must be carried out as described in “Machine Settings for Initial Commissioning by the Service Engineer”.

### **17.1.2 Observe the environmental protection regulations**



Legal obligations relating to the avoidance of waste materials and to their recycling/removal in accordance with applicable regulations must be observed!

In particular, during installation, repair and maintenance work, materials that could pollute water such as: Grease and oils, Cleaning fluids containing solvents, must not pollute the ground or run into the sewerage system! These materials must be stored, shipped, collected and disposed of in suitable containers!

## **17.2 Dosing units**

The dosing units themselves are maintenance free in principle but the working life is largely dependent on the chemical used.

### **17.2.1 Change of products**

Change of product means that one rinse aid or detergent product is replaced by another. The use of differing products alongside each other can result in break-downs.

- Hose lines and dosing units must always be rinsed out with warm water.

## 17.3 Maintenance plan



### NOTE

Maintenance work should only be conducted by authorised MEIKO personnel.

Maintenance procedures	FV 28G / FV28GIO EcoStar 430 F EcoStar 530 F-M	FV 40.2 / FV 40.2 G /FV 60.2 / FV 70.2 D TopClean 60	GK 60	OR 50 H	EcoStar 545D / DV 80.2 / DV 120.2 / DV 125.2 / DV 200.2 / DV 200.2 PW	DV 270 B	FV 130.2 – FV 250.2 / DV 270.2	Component OK	Component faulty	Component
<b>1. Pumps</b>										
Check pumps for watertightness, pump rotor noise, rotation direction and function										
Check pump suction										
Check pump sieves correctly fitting and operating correctly										
Check sliding ring washer/contra-rotation ring										
<b>2. Wash systems</b>										
Check water level in tank										
Check that wash water pipe is watertight										
Check cleaning system is complete and produces correct spray pattern										
Check wash arm hubs										
<b>3. Fresh water rinse</b>										
Check flow pressure/water pressure										
Check rinsing system is complete and produces correct spray pattern										
Check that system is watertight										
<b>4. Housing and mounting parts</b>										
Check housing, tank, sheet metal cover, hood, doors and covering of machine base for damage and correct operation										
Check tank cover sieves										
Check boiler, hoses, clamps, plastic parts and seals										
Check operation of raising and lowering equipment										
<b>5. Fresh water installation</b>										
Check level regulation										
Check valves, clean dirt trap										
Check that all connections (incl. hand spray) are watertight										
Check settings of built-in water softener (if fitted)										
Check operation of complete or partial water softener (if fitted)										
<b>For the GiO module:</b> Perform pre-filter change (must be completed every 6 months at the latest)										
Check water hardness										
<b>6. Waste water equipment</b>										
Check if watertight										
Check pressure hose position and operation of drain pump										
<b>7. Electrical installation</b>										
Check of all fuses										
Tighten all electrical connections										
Check tank and boiler heating										
Check thermostat and stop switch										
<b>8. Electrical safety check (certificate is optional)</b>										



<b>Maintenance procedures</b>	FV 28G / FV28GIO EcoStar 430 F EcoStar 530 F-M	FV 40.2 / FV 40.2 G /FV 60.2 / FV 70.2 D TopClean 60	GK 60	OR 50 H	EcoStar 545D / DV 80.2 / DV 120.2 / DV 125.2 / DV 200.2 / DV 200.2 PW	DV 270 B	FV 130.2 – FV 250.2 / DV 270.2	Component OK	Component faulty	Component
Visual inspection								at least once a year		
Protective conductor check								at least once a year		
Measure insulation resistance								at least once a year		
Protection conductor current measurement								at least once a year		
<b>9. Detergent dosing</b>										
Check dosage, adjust if necessary										
<b>10. Rinse aid dosing</b>										
Check dosage, adjust if necessary										
<b>11. Operation check of the complete machine</b>										
Check machine for correct interaction of all functions										
<b>12. Test run</b>										
Check results of test wash and rinse										
Brief instruction for new personnel										

## 18 Dismantling and disposal

In addition to valuable raw materials and recyclable materials, the packaging and the old device may also contain substances that are harmful to health and the environment and were required for the function and safety of the old device.

Please do not dispose of your old device in residual waste. Instead, contact your dealer or the collection points set up in your community for information regarding the disposal of your old device.

### 18.1 Disposal of packaging materials

All the packaging materials are recyclable. The following materials are used:

- Square timber frame
- Plastic sheeting (PE film)
- Cardboard packaging (edge protection)
- Packaging strap (steel strip)
- Packaging strap (plastic (PP))



#### Note

The square timber frame consists of untreated raw pine / spruce. In order to guard against pests, country-specific import regulations may also stipulate the use of treated wood.

### 18.2 Dismantling and disposal of the old device

#### Warning



#### Risk of injury from contact with chemicals

Detergent and rinse aid result in damage to health if in contact with skin or eyes or if swallowed.

- Use eye protection.
  - Wear protective gloves.
  - Contact a physician immediately if chemicals or water containing chemicals (wash water) are swallowed.
- 
- Where appropriate, rinse machine components, containers, dosing units and hoses with fresh water to remove chemical residues. Wear suitable clothes (gloves, safety glasses) for this.



The device is marked with this symbol. Please observe the local regulations for proper disposal of your old device.

The components should be separated by material for recycling.

## 19 Documentation

Installation drawing / technical sheet

Technical Data

Wiring diagram / Programming instructions





**MEIKO Maschinenbau GmbH & Co. KG**

Englerstraße 3

77652 Offenburg

Germany

[www.meiko-global.com](http://www.meiko-global.com)

[info@meiko-global.com](mailto:info@meiko-global.com)