

MEIKO TopCut B

Cuts open and empties urine and secretion bags in
MEIKO washer-disinfectors



MEIKO TopCut B

Make your MEIKO washer-disinfector even more 'cutting-edge'!

SAFE TO USE, GREAT FOR CUTTING COSTS

The *TopCut B* accessory makes your MEIKO washer-disinfector even more versatile by adding the ability to empty urine and secretion bags. One moment the machine is cleaning and disinfecting care utensils – the next, it is cutting, emptying and disinfecting bags of thin secretions. Switching between these tasks is easy thanks to the quick-change system. This device can be used with all MEIKO washer-disinfectors, whether they have a manual or automatic door (optional).



BENEFITS

- Reduces volume of healthcare waste
- Cuts cost of waste disposal
- Keeps staff and patients safe – bags are emptied in a closed system
- Disinfection carried out inside the machine
- Blade is kept safely away from users
- Low-maintenance, wear-resistant blade

GETS THE JOB DONE

TopCut B is a safe and reliable system that makes life easier for users. The built-in blade only emerges once the wash chamber door is almost completely closed. As soon as you open the door, the blade returns automatically and safely to its original position.

- Safe and hygienic way to empty bags
- Placing bags in the machine is quick and easy
- Can be used with many other types of bag – contact us to find out more

HOW IT WORKS



1. Remove the standard care utensil rack



2. Insert the *TopCut B* thanks to the clever quick-change system



3. Place the urine or secretion bag in the rack



4. Cutting starts automatically as soon as you close the door

Disposing of waste is expensive – less waste means lower costs

