

TopCut B

The safest way to empty bags

TopCut B – even more flexible with regard to safety, hygiene and cost-effectiveness: The innovative cutting and opening device for urine and secretion bags in MEIKO bedpan washer-disinfectors.



TopCut B

A cutting and opening device which can quickly and easily replace the standard utensil holders. Safe and cost-effective opening, emptying and disinfection of urine and secretion bags.

Extremely safe and economical


Your MEIKO bed-pan cleaner achieves exceptional flexibility by the addition of the **TopCut B** cutting and opening device. It takes no time at all to convert your TopLine appliance from cleaning and disinfecting care utensils to become an economic means of opening, emptying and disinfecting bags filled with urine or thin secretions. The cutting process which opens the bags occurs in a sealed environment and is therefore absolutely hygienic and safe for both staff and patients.

- Reduction in the amount of refuse
- Reduction in refuse disposal charges
- Reliable cutting technique
- Safety and security for staff and patients as the bags are opened in a closed system
- Disinfection inside the appliance
- Effective protection against injury from the blade
- Low wear and maintenance blade



This is how the **TopCut B** cutting and opening device transforms your MEIKO bedpan washer-disinfector quickly and easily into a first-class appliance for opening and disinfecting urine and secretion bags.

Cutting waste disposal fees
by reducing the amount of waste

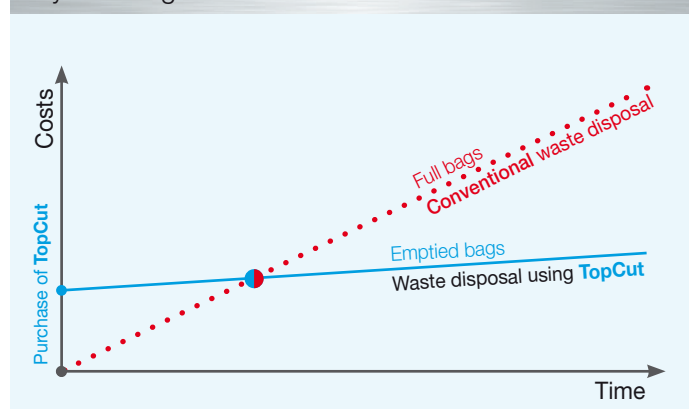

1 Just take out the racks for normal care utensils



2 Fit **TopCut B** with the aid of an intelligent rapid exchange system



3 Insert the urine or secretion bag



4 Close the door and the bag is opened automatically

Simply number 1 in practice

TopCut B's uncomplicated and reliable operation in day-to-day work never fails to impress.

- The bags are quickly and easily inserted
- The blade remains behind its guard until the door is almost closed, thus ensuring that the bag is only opened under safe and hygienic conditions
- The blade automatically returns to its initial position behind the guard when the door is opened
- Many different types of bags can be processed – just ask for details

