

MEIKO TopCut H

Versatile rack as cutter for emptying urine and secretion bags
and holder for care utensils



MEIKO TopCut H

Make your MEIKO washer-disinfector even more 'cutting-edge'!

SAFE TO USE, GREAT FOR CUTTING COSTS

By combining a universal care utensil holder with a built-in cutter for emptying urine and secretion bags, the *TopCut H* makes your MEIKO washer-disinfector even more versatile:

- Perfect for cleaning and disinfecting commonly used care utensils...
- ...and equally good for cutting open, emptying and disinfecting bags containing urine or other thin secretions.

No need to switch: the same holder can be used to empty, clean and disinfect care utensils and various types of bag. Compatible with all MEIKO washer-disinfectors, whether they have a manual or automatic door (optional).

BENEFITS

- Greater flexibility
- Reduces volume of healthcare waste
- Cuts cost of waste disposal
- Keeps staff and patients safe – bags are emptied in a closed system
- Disinfection carried out inside the machine
- Blade is kept safely away from users

GETS THE JOB DONE

TopCut H is a safe and reliable system that makes life easier for users. The built-in blade only emerges once the wash chamber door is almost completely closed. As soon as you open the door, the blade returns automatically and safely to its original position.

- Safe and hygienic way to empty bags
- Placing bags in the machine is quick and easy
- Can be used with many other types of bag – contact us to find out more



TopCut H as holder

TopCut H as bag-emptying device



The *TopCut H* is compatible with all standard care utensils and multiple bag types

Disposing of waste is expensive – less waste means lower costs

