

Short operating instructions TopCut B (Bag to 5l)




	⚠ DANGER
	<p>Cuts to fingers and hands!</p> <p>Do not activate the blade cover manually if the TopCut is not installed.</p> <p>The TopCut has a sharp blade at the front. In idle position, the blade is covered by the cutting unit and is not exposed until the washing chamber door is closed.</p>

Insert TopCut






Hold TopCut with the right hand.
Move the basket in the direction of the arrow with your left hand.

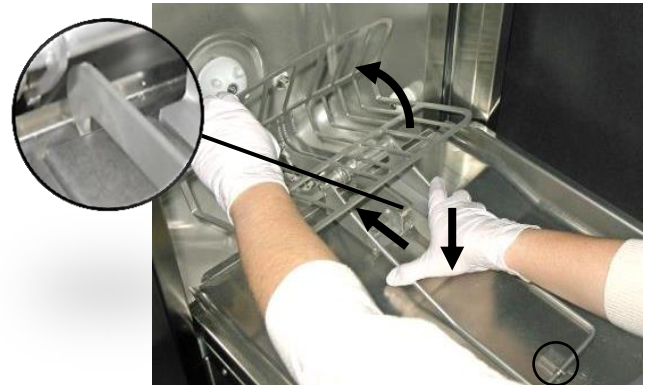

Press down the lever with your right-hand palm and simultaneously push the bracket forwards until the lever engages in front of the retaining shoe. Move the basket in the direction of the arrow. *The TopCut is ready for use.*

Inserting a bag

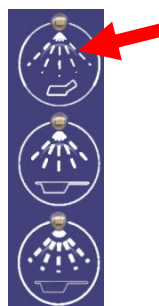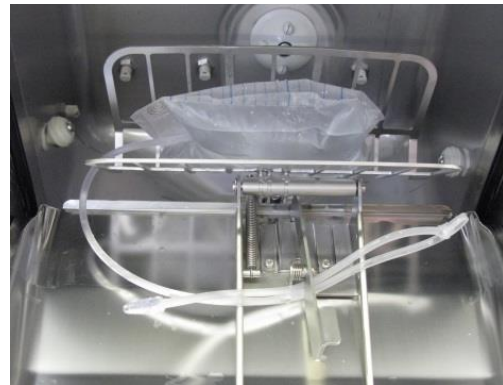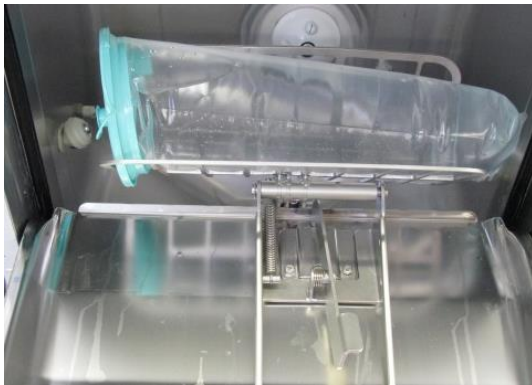

Close the door - bag is cut open automatically

Start the washing program - select **short program**

After the program has been completed, remove the empty bag, which has been disinfected from the outside, and dispose of it.