

Professional hygiene technology

MEIKO TopLine

Washer-disinfectors



MEIKO Care



RELIABLE HYGIENE MANAGEMENT FOR DIRTY UTILITY ROOMS

Legal compliance, duty of care, staff health and safety, and the constant monitoring and documentation of infection control measures – these are the non-negotiables of modern care, wherever you operate. The goal is clear: preventing infection in hospitals and care facilities.

MEIKO provides the expertise and reliable cleaning and disinfection technology you need to achieve this.

Whatever your needs, our bespoke hygiene management solutions have got you covered. Meeting

statutory and infection control requirements is a serious business – but with MEIKO at your side you can be confident of getting it right.

From effective reprocessing of care utensils to reliable, user-friendly operation, the *TopLine* range features impressive technology that ensures the safety of both patients and staff while adhering strictly to the requirements of EN ISO 15883.




KEY FEATURES:

- Premium technology – hygiene standards patients can trust
- Steamtight system – a safe working environment for your staff
- Maximum flexibility – freely selectable A_0 values (60, 600, 3,000) and a multifunctional holder concept for different care utensils
- Safe and hygienic – filtered-air drying and cooling with system disinfection of all water lines
- MEIKO 360° – the complete solution including chemicals, service, the MEIKO Academy and planning

MEIKO TopLine – Technology

One of MEIKO's most powerful washer-disinfectors, featuring integrated cooling, system disinfection, telescopic rotary nozzle and much more. Designed to stop outbreaks at source in the dirty utility room.

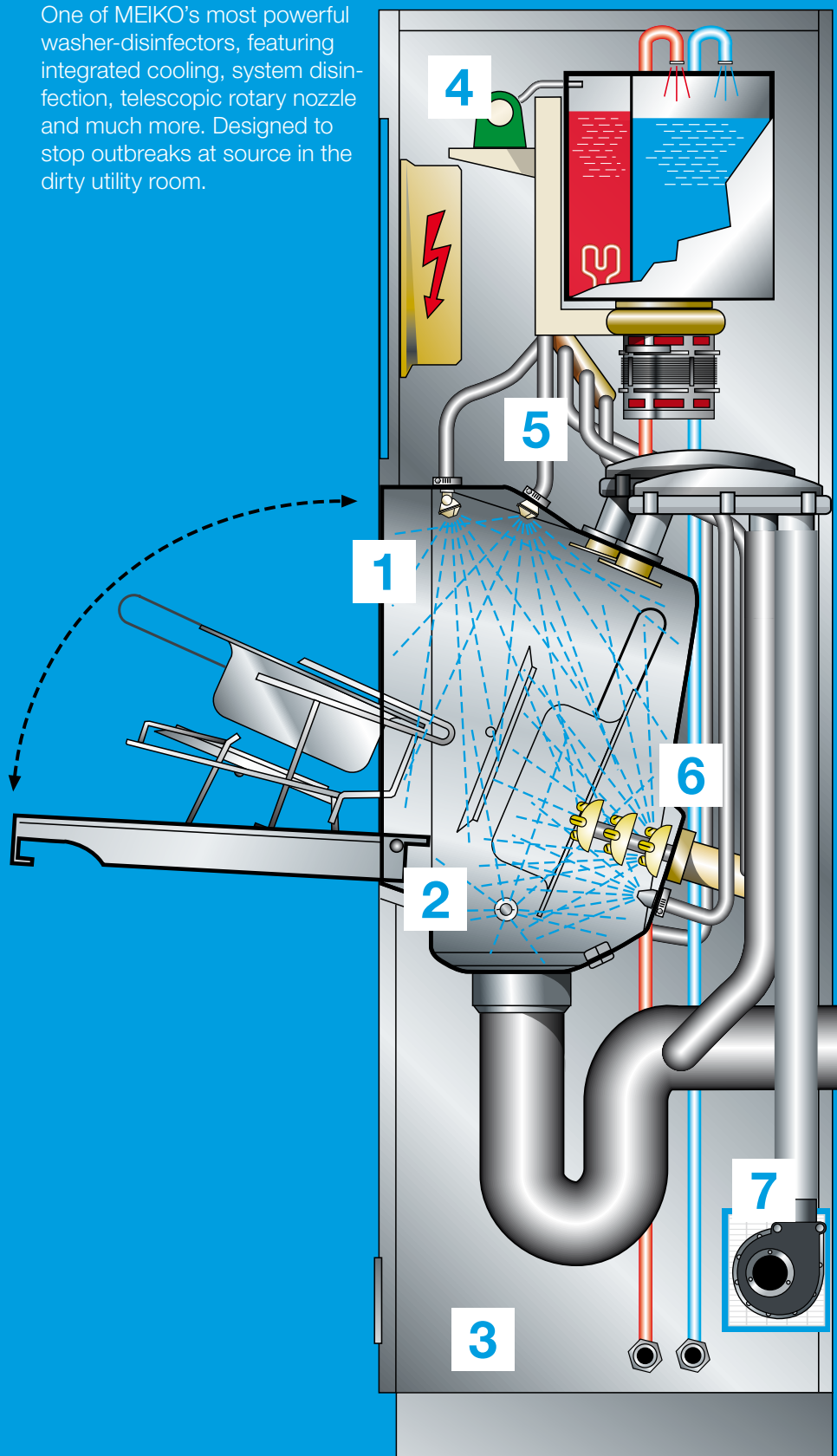



Steam
Stop

Thermo
Control

Chem
Code

Chem
Control

System
Disinfect+

Power
Clean+

Cool
Touch

KEY FEATURES:

1. A fully steamtight system that controls and safely releases steam

Benefits

- Effectively prevents the release of germs
- Enhances workplace safety and hygiene
- Reduces moisture damage to the appliance and the room

2. Two temperature sensors that monitor each other, taking measurements exactly where it counts – at the coldest point

Benefits

- Monitors compliance with A_0 values
- Ensures reliable disinfection performance
- Meets the requirements of EN ISO 15883-1 and -3

3. Colour- and shape-coded chemical connections for rinse aid with water softener and for detergent

Benefits

- Reliably prevents mix-ups
- Ensures clear workflows
- Makes day-to-day operation easier

4. Sensor-controlled monitoring of the dosing volume and function for rinse aid with water softener and for detergent

Benefits

- Optimises the use of rinse aid with water softener and detergent
- Ensures process reliability in accordance with EN ISO 15883-1 and -3
- Prevents limescale build-up and resulting damage

5. System disinfection of the wash chamber, water-carrying pipes, nozzles and water tank during every cycle

Benefits

- Meets the current standard requirements according to EN ISO 15883-1
- Reliably prevents recontamination
- Ensures consistent disinfection performance through uniform steam distribution

6. Cleaning system with telescopic rotary nozzle and additional optimally positioned wash nozzles

Benefits

- Increases the effectiveness of internal cleaning, even with a wide variety of bedpan shapes
- Effectively removes germs and actively flushes them away
- Effective even against C. diff spores (confirmed by expert opinion) thanks to the combination of mechanical action, temperature and detergent

7. Recooling with filtered air – including drying effect

Benefits

- Cools with germ-free air thanks to HEPA filtration
- Ensures care utensils are immediately ready for use again
- Reduces the risk of recontamination compared to water-based cooling

MEIKO TopLine – Machine upgrades and settings



AUTOMATIC DOOR WITH INFRARED HAND SENSOR AND/OR FOOT SWITCH: (optional)

- Minimises the risk of pathogen spread
- Easier to use in care settings

ADJUSTABLE A₀ VALUE:

- Flexibly choose your A₀ value
- Available settings: A₀ 60, A₀ 600, A₀ 3,000
- Set in consultation with your infection control team, for basic hygiene or as required

THE PERFECT A₀ VALUE IN THREE STEPS



Step 1:

Choose the cleaning programme based on the type of care utensils and the degree of soiling – press the button and hold it down.




Step 2:

Display shows the available A₀-settings: A₀ 60 / A₀ 600 / A₀ 3,000. Release the button when the setting you want appears in the middle.



Step 3:

The programme starts automatically as soon as you release the button.

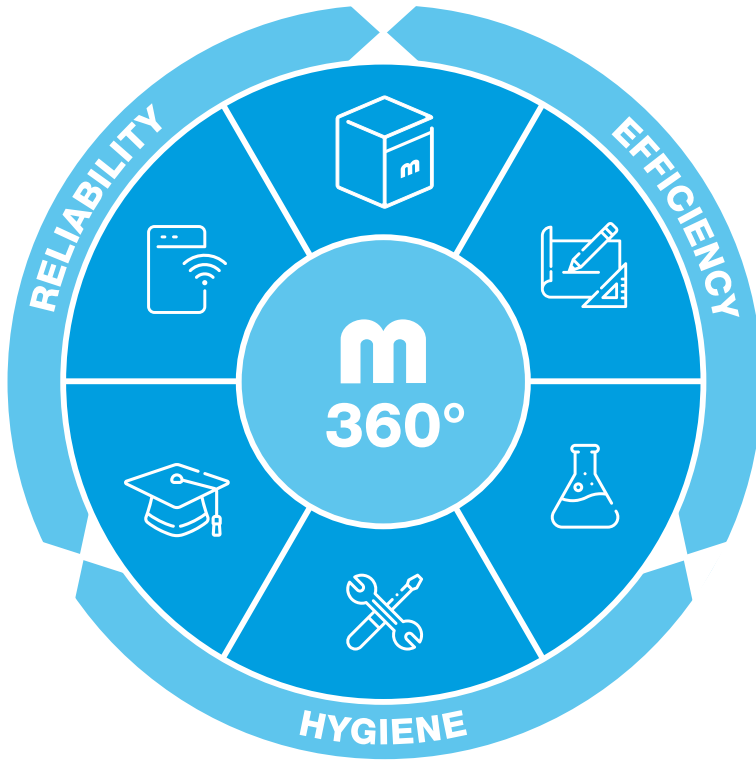

**MEIKO 360°: HOLISTIC BY DESIGN.
EXPERTLY DELIVERED.**

Perfect cleanliness is never an accident. It's the result of everything working in harmony. MEIKO 360° combines all the critical factors in one integrated solution: from expert planning, high-performance technology and perfectly matched chemicals to reliable service, smart connectivity and knowledge transfer through the MEIKO Academy.

Together with you, we forge many separate parts into one powerful whole – hygienic, efficient and reliable.

MEIKO CHEMICALS

To achieve outstanding cleaning results, you need exactly the right combination of technology, detergent (optional) and rinse aid. That's what you get with Doyen products approved by MEIKO.

Benefits of Doyen chemicals:

- Approved for use in TopLine appliances
- Optimal dosing quantity, kept as low as possible
- Tested for material compatibility



MEIKO TopLine – Everything from a single source



MEIKO PLANNING

Whether you need an individual appliance or a combined care unit, we'll work together with you to plan your entire utility room. That's the best way to create outstandingly hygienic, ergonomically optimised work spaces that meet both customer expectations and regulatory requirements.

Our hallmarks include:

- Tailor-made clean solutions for refits, renovations and new-build projects
- Seamless connections
- Flush surfaces that integrate individual components perfectly
- Stainless steel throughout



MEIKO ACADEMY

Did you know that we also offer training for in-house technicians? The MEIKO Academy provides direct access to the knowledge we've built up over decades. Our online and in-person training will bring you up to speed on the latest technology. The training courses also cover key topics from the world of microbiology and infection control, as well as important legal issues.



Find out more about our training programmes at:
www.meiko-academy.com



MEIKO SERVICE

Our after-sales service teams offer a rapid response, fast arrival times and service vans stocked with plenty of spare parts.

Benefits of MEIKO Service:
24/7 service hotline, 365 days a year
Authorised service technicians trained at the
MEIKO Academy.



MEIKO TopLine – Model overview

Individual appliances

TopLine 10 – Wall-mounted model (with or without slop sink)
Wall mounting keeps the floor clear for easy cleaning



Dimensions (in mm):
W 500, D 450 or 588, H (appliance) 1,630



Dimensions (in mm):
W 1,000, D 450 or 588, H (appliance) 1,630

TopLine 20 – Floor-standing model (with or without slop sink)
Floor mounting with plinth for rapid installation



Dimensions (in mm):
W 500, D 450 or 588, H 1,730



Dimensions (in mm):
W 1,000, D 450 or 588, H 1,730

Images show optional features/accessories

TopLine 30 – Flush-mounted model (with or without WC connection)
For flush-mounted installation in ensuite bathrooms

Dimensions (in mm):
Base frame:
W 650, D 450, H 2,210
Visible fronts:
W 530, H 1,760



Dimensions (in mm):
Base frame:
W 650, D 450, H 2,210
Visible fronts:
W 530, H 1,760

TopLine 20 – Table top unit (with or without hand wash basin)
Floor-mounted with plinth to maximise work space and storage surface



Dimensions (in mm):
Appliance:
W 900, D 600, H 900



Dimensions (in mm):
Appliance:
W 900, D 600, H 1,000
Hand wash basin:
W 340, D 370, H 150

MEIKO TopLine – Combined care units

MEIKO combined care units are available in a range of configurations, with floor-standing units or table top units tailored to your specific requirements and utility room layout. Offering maximum functionality, they help make day-to-day care work easier.

SAN 14 B / MEIKO TopLine

Floor-standing or table top unit with slop sink unit and hand wash basin



Dimensions (mm): W 1,400, D 600, H 900
Slop sink unit: H 700
Slop sink basin: Ø 362



Dimensions (mm): W 1,400, D 600, H 1,000
Slop sink unit: H 700
Slop sink basin: Ø 362 mm
Hand wash basin: W 340, D 370, H 150

SAN 19 B / MEIKO TopLine

Floor-standing or table top unit with slop sink unit, hand wash basin and cabinet (500 mm)

Dimensions (mm): W 1,900, D 600, H 900
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150
Cabinet space: W 500



SAN 20 BW / MEIKO TopLine

Floor-standing or table top unit with slop sink unit, hand wash basin, wash basin and cabinet (600 mm)

Dimensions (mm): W 2,000, D 600, H 1,000
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150
Wash basin: W 500, D 400, H 300
Cabinet space: W 600



SAN 24 W / MEIKO TopLine

Floor-standing or table top unit with slop sink unit, hand wash basin and cabinet (1,000 mm)

Dimensions (mm): W 2,400, D 600, H 900
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 500, D 400, H 300
Cabinet space: W 1000



SAN 29 BW / MEIKO TopLine

Floor-standing or table top unit with slop sink unit, hand wash basin, wash basin and cabinet (1,500 mm)

Dimensions (mm): W 2,900, D 600, H 1,000
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150
Wash basin: W 500, D 400, H 300
Cabinet space: W 1,500



MEIKO TopLine – Accessories

In addition to its cleaning and disinfection technology, MEIKO offers a comprehensive range of accessories and fittings. These promote safe, ergonomic working conditions and enable hygienic storage of care utensils. We are also happy to develop customised solutions tailored to your needs!



Drying and storage racks*

- For disinfected urine bottles, bedpans and containers with storage grid or drip tray
- Available in widths from 500 to 1,400 mm



Wall shelving*

- With height-adjustable shelves
- Available in all widths (depth 350 mm)



Closed wall-mounted cupboards**

- With hinged doors and one intermediate shelf
- Available in widths 500/600/800/1,000/1,200/1,500 mm
- Optional: light screen and lamp



Open wall-mounted cupboards

- For open storage with one intermediate shelf
- Available in widths 500/1,000/1,200 mm



Cleaning combination*

- Base cabinets with drawers and other features for combined care units
- Available in a variety of widths and heights




Slop sink unit*

- Standard model optionally equipped with integrated slop sink flushing
- Can be operated independently of the washer-disinfector


* Images show optional features/accessories

** Optional: light screen and lamp


MEIKO TopLine floor-standing unit

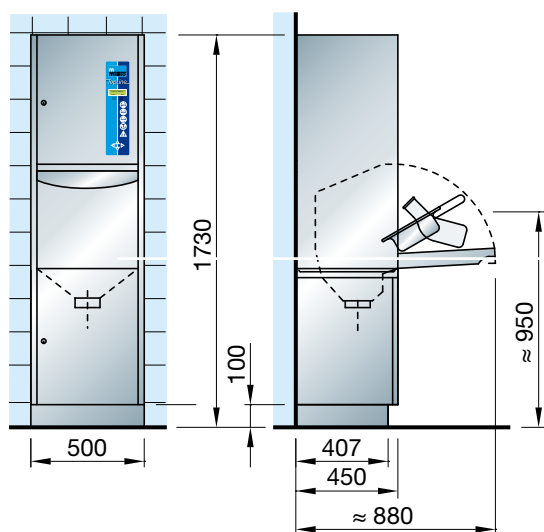

MEIKO TopLine small combined care unit

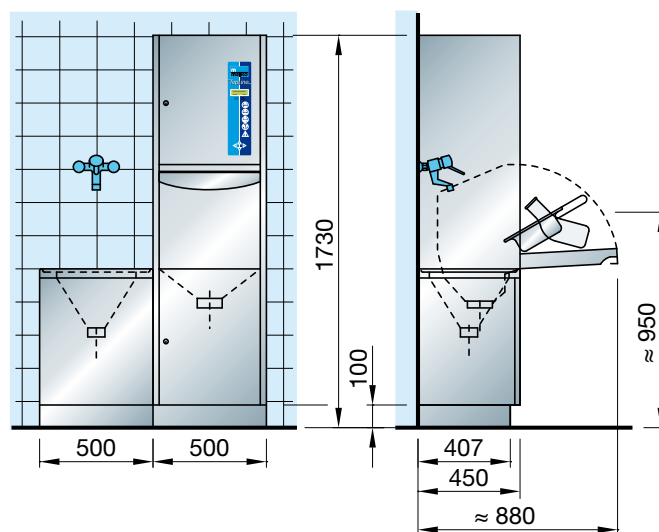

MEIKO TopLine cabinet model

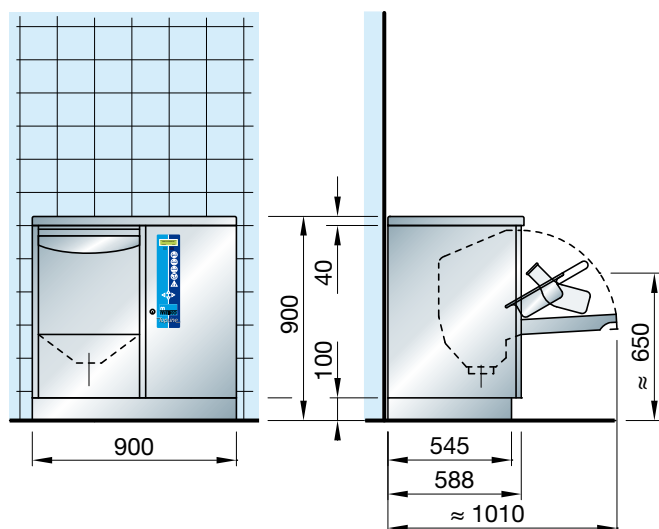

		TopLine	
		400 V	230 V
Total connected load	kW	4.6	2.8
Feed water connection, cold water		G 3/4"	
Feed water connection, hot water 45-60 °C		G 3/4"	
Flow pressure	bar	1-5	
Flow rate	l/min	18	
Drain		DN 100	
Noise level	dB(A)	48.2	



The clean solution

Manufacturer:
MEIKO Maschinenbau GmbH & Co. KG · Englerstr. 3 · 77652 Offenburg, Germany
Phone +49 781 203-0 · info@meiko-global.com · www.meiko-global.com



720.031.02.06.26/EN/

We reserve the right to make changes
as part of product development.