

Professional hygiene technology

MEIKO TOPIC

Washer-disinfectors





MORE TIME FOR PEOPLE

At the touch of a button, MEIKO washer-disinfectors clean and disinfect your care utensils. They reliably eliminate bacteria, viruses, and even *C. diff* spores, in full compliance with EN ISO 15883.

The result is outstanding hygiene – plus a lighter workload for your care staff.

The benefits also include improved operational efficiency. Jörg Menzel, Managing Director of Blindenwohnstätten Berlin – a care home for blind people – explains why that is so important: "Care facilities like ours need access to technology that reduces our workload and makes processes more efficient."

Whatever your layout, *TOPIC* adapts to the available space. Choose between a floor-standing unit, table top model or wall-mounted version with floor clearance. We can even provide it as a fully equipped combined care unit, delivering everything you need for smooth day-to-day operations.

Individual *TOPIC* appliances are designed for quick, hassle-free replacement of outdated equipment. They slot perfectly into the exact footprint of your old machine – regardless of the original make – using your existing connections.



KEY FEATURES:


- Reliable hygiene thanks to A_0 value control in accordance with EN ISO 15883
- Steamtight cleaning and disinfection system
- Three effective programme cycles (short, normal, intensive)
- Easy operation with One Touch controls
- Cost savings through low resource consumption
- Flexibility thanks to multifunctional holders
- MEIKO 360° – the complete solution including chemicals, service, the MEIKO Academy and planning

MEIKO TOPIC – Technology

Top-of-the-line tech made from non-rusting stainless steel lays the groundwork for durability, making your investment an especially safe bet.

Best of all: the deep-drawn wash chamber is designed without corners or edges, making it ideal for self-cleaning. The highly efficient, optimally positioned power nozzle system ensures optimal cleaning of urine bottles, bed-pans, commode buckets and more.

One-Touch technology makes programme control effortless. From programme start to analysis, everything is operated via the membrane keyboard.

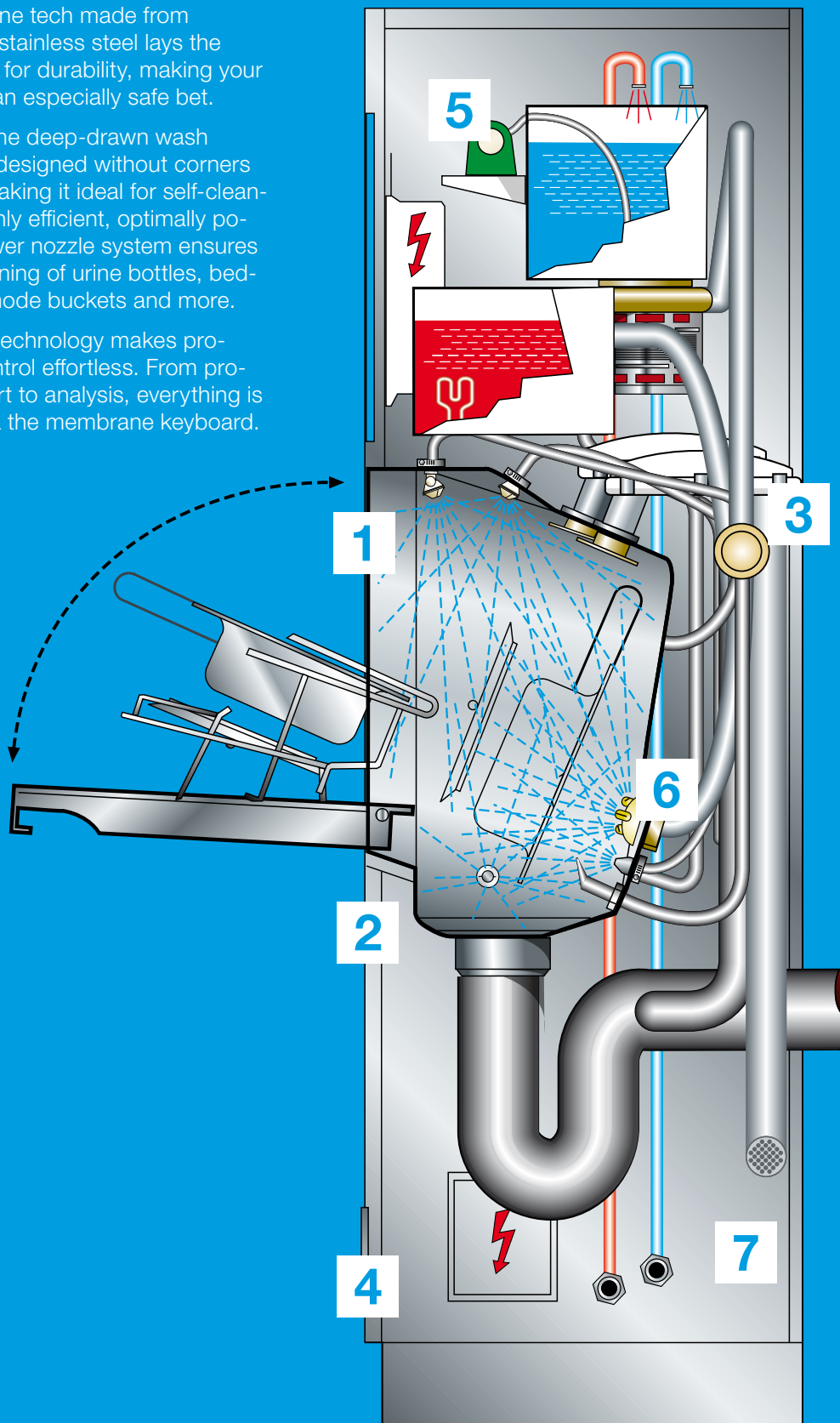



Steam
Stop

Thermo
Control

System
Disinfect

Chem
Code

Chem
Control

Power
Clean

Cool
Touch

KEY FEATURES:

1. A fully steamtight system that controls and safely releases steam

Benefits

- Effectively prevents the release of germs
- Enhances workplace safety and hygiene
- Reduces moisture damage to the appliance and the room

2. Two temperature sensors that monitor each other, taking measurements exactly where it counts – at the coldest point

Benefits

- Monitors compliance with A_0 values
- Ensures reliable disinfection performance
- Meets the requirements of EN ISO 15883-1 and -3

3. System disinfection of the wash chamber, water-carrying pipes and nozzles during every cycle

Benefits

- Reliably prevents recontamination
- Ensures consistent disinfection performance through uniform steam distribution

4. Colour- and shape-coded chemical connections for rinse aid with water softener and for detergent

Benefits

- Reliably prevents mix-ups
- Ensures clear workflows
- Makes day-to-day operation easier

5. Sensor-controlled monitoring of the dosing volume and function for rinse aid with water softener and for detergent

Benefits

- Optimises the use of rinse aid with water softener and detergent
- Ensures process reliability in accordance with EN ISO 15883-1 and -3
- Prevents limescale build-up and resulting damage

6. Cleaning system with a rotary nozzle and additional optimally positioned wash nozzles

Benefits

- Enables thorough cleaning with minimal time and resources
- Effectively removes germs and actively flushes them away
- Effective even against *C. diff* spores (confirmed by expert opinion) thanks to the combination of mechanical action, temperature and detergent

OPTION

7. Recooling with filtered air – including drying effect

Benefits

- Cools with germ-free air thanks to HEPA filtration
- Ensures care utensils are immediately ready for use again
- Reduces the risk of recontamination compared to water-based cooling

MEIKO TOPIC – Machine upgrades and settings



AUTOMATIC DOOR WITH INFRARED HAND SENSOR AND/OR FOOT SWITCH: (optional)

- Minimises the risk of pathogen spread
- Easier to use in care settings

ADJUSTABLE A₀ VALUE:

- Flexibly choose your A₀ value
- Available settings: A₀ 60, A₀ 600, A₀ 3,000 (3,000 when combined with CoolTouch)
- Set in consultation with your infection control team, for basic hygiene or as required

THE PERFECT A₀ VALUE IN THREE STEPS



Step 1:

Choose the cleaning programme based on the type of care utensils and the degree of soiling – press the button and hold it down.




Step 2:

Display shows the available A₀-settings: A₀ 60 / A₀ 600 / A₀ 3,000. Release the button when the setting you want appears in the middle.



Step 3:

The programme starts automatically as soon as you release the button.

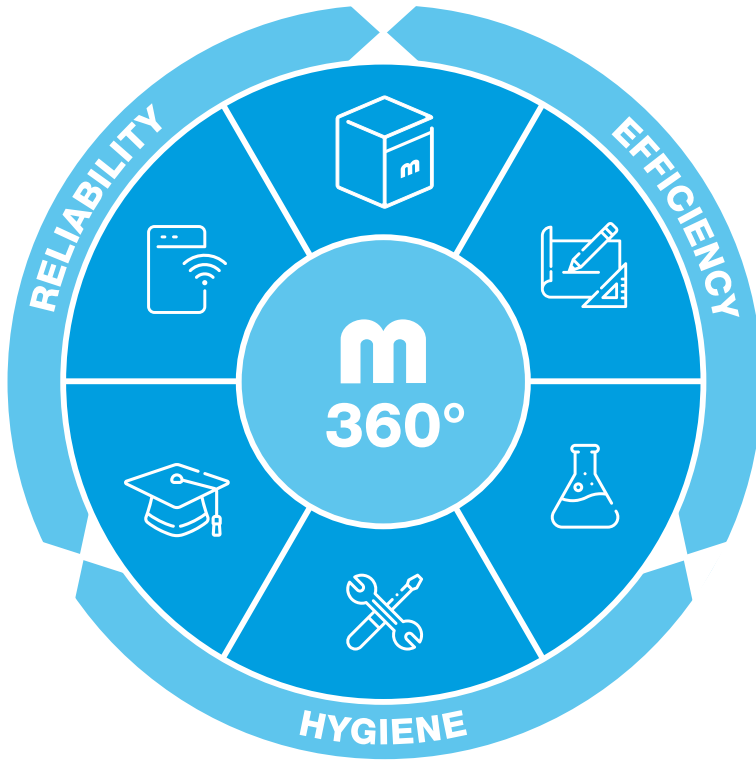

MEIKO 360°: HOLISTIC BY DESIGN. EXPERTLY DELIVERED.

Perfect cleanliness is never an accident. It's the result of everything working in harmony. MEIKO 360° combines all the critical factors in one integrated solution: from expert planning, high-performance technology and perfectly matched chemicals to reliable service, smart connectivity and knowledge transfer through the MEIKO Academy.

Together with you, we forge many separate parts into one powerful whole – hygienic, efficient and reliable.

MEIKO CHEMICALS

To achieve outstanding cleaning results, you need exactly the right combination of technology, detergent (optional) and rinse aid. That's what you get with Doyen products approved by MEIKO.

Benefits of Doyen chemicals:

- Approved for use in TOPIC appliances
- Optimal dosing quantity, kept as low as possible
- Tested for material compatibility



MEIKO TOPIC – Everything from a single source



MEIKO PLANNING

Whether you need an individual appliance or a combined care unit, we'll work together with you to plan your entire utility room. That's the best way to create outstandingly hygienic, ergonomically optimised work spaces that meet both customer expectations and regulatory requirements.


Our hallmarks include:

- Tailor-made clean solutions for refits, renovations and new-build projects
- Seamless connections
- Flush surfaces that integrate individual components perfectly
- Stainless steel throughout



MEIKO ACADEMY

Did you know that we also offer training for in-house technicians? The *MEIKO Academy* provides direct access to the knowledge base we have built up over decades. Our online and in-person training will bring you up to speed on the latest technology. The training courses also cover key topics from the world of microbiology and infection control, as well as important legal issues.

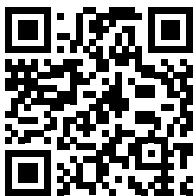

Learn more about our training programmes at:
www.meiko-academy.com



MEIKO SERVICE

Our after-sales service teams offer a rapid response, fast arrival times and service vans stocked with plenty of spare parts.

Benefits of *MEIKO Service*: 24/7 service hotline, 365 days a year. Authorised service technicians trained at the *MEIKO Academy*.



MEIKO TOPIC – Model overview

Individual appliances

MEIKO TOPIC 10 – Wall-mounted unit (with or without slop sink unit)

Wall mounting keeps the floor clear for easy cleaning



MEIKO TOPIC 10

Dimensions (mm): W 500, D 450 or 588, H (appliance) 1,630



MEIKO TOPIC 10 A

Dimensions (mm): W 1,000, D 450 or 588, H (appliance) 1,630
Splash guard (optional)

MEIKO TOPIC 20 – Floor-standing unit (with or without slop sink unit)
Floor mounting with plinth for rapid installation



MEIKO TOPIC 20
Dimensions (mm): W 500, D 450 or 588, H 1,730



MEIKO TOPIC 20 A
Dimensions (mm): W 1,000, D 450 or 588, H 1,730
Splash guard (optional)

MEIKO TOPIC – Model overview

Individual appliances

MEIKO TOPIC 40 – Table top unit (with or without hand wash basin)

Floor-mounted with plinth to maximise work space and storage surface

- Equipped as standard with a lockable storage cabinet for chemical containers
- Tabletop available with upstand for an additional charge



MEIKO TOPIC 40
Dimensions (mm): W 900, D 600, H 900



MEIKO TOPIC 40 B
Dimensions (mm): W 900, D 600, H 1,000
Hand wash basin: B 340, T 370, H 150

Combined care units

MEIKO combined care units are available in a range of configurations, with floor-standing units or table top units tailored to your specific requirements and layout. Offering maximum functionality, they help make day-to-day care work easier.

SAN 14 B / MEIKO TOPIC

Floor-standing or table top unit with slop sink unit and hand wash basin

Dimensions (mm): W 1,400, D 600, H 1,000
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150



SAN 19B – Floor-standing or table top unit with slop sink unit, hand wash basin and cabinet (500 mm)

Dimensions (mm): W 1,900, D 600, H 900
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150
Cabinet space: W 500



SAN 20BW – Floor-standing or table top unit with slop sink unit, hand wash basin, wash basin and cabinet (600 mm)

Dimensions (mm): W 2,000, D 600, H 1,000
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150
Wash basin: W 500, D 400, H 300
Cabinet space: W 600



SAN 24W – Floor-standing or table top unit with slop sink unit, wash basin and cabinet (1,000 mm)

Dimensions (mm): W 2,400, D 600, H 900
Slop sink unit: H 700
Slop sink basin: Ø 362
Wash basin: W 500, D 400, H 300
Cabinet space: W 1,000



SAN 29BW – Floor-standing or table top unit with slop sink unit, hand wash basin, wash basin and cabinet (1,500 mm)

Dimensions (mm): W 2,900, D 600, H 900
Slop sink unit: H 700
Slop sink basin: Ø 362
Hand wash basin: W 340, D 370, H 150
Wash basin: W 500, D 400, H 300
Cabinet space: W 1,500



MEIKO TOPIC – Accessories

In addition to its cleaning and disinfection technology, MEIKO offers a comprehensive range of accessories and fittings. These promote safe, ergonomic working conditions and enable hygienic storage of care utensils. We are also happy to develop customised solutions tailored to your needs!



Drying and storage racks*

- For disinfected urine bottles, bedpans and containers with storage grid or drip tray
- Available in widths from 500 to 1,400 mm



Wall shelving*

- With height-adjustable shelves
- Available in all widths (depth 350 mm)



Closed wall-mounted cupboards**

- With hinged doors and one intermediate shelf
- Available in widths 500/600/800/1,000/1,200/1,500 mm
- Optional: light screen and lamp



Open wall-mounted cupboards

- For open storage with one intermediate shelf
- Available in widths 500/1,000/1,200 mm



Cleaning combination*

- Base cabinets with drawers and other features for combined care units
- Available in a variety of widths and heights




Slop sink unit*


- Standard model optionally equipped with integrated slop sink flushing
- Can be operated independently of the washer-disinfector

* Images show optional features/accessories


** Optional: light screen and lamp

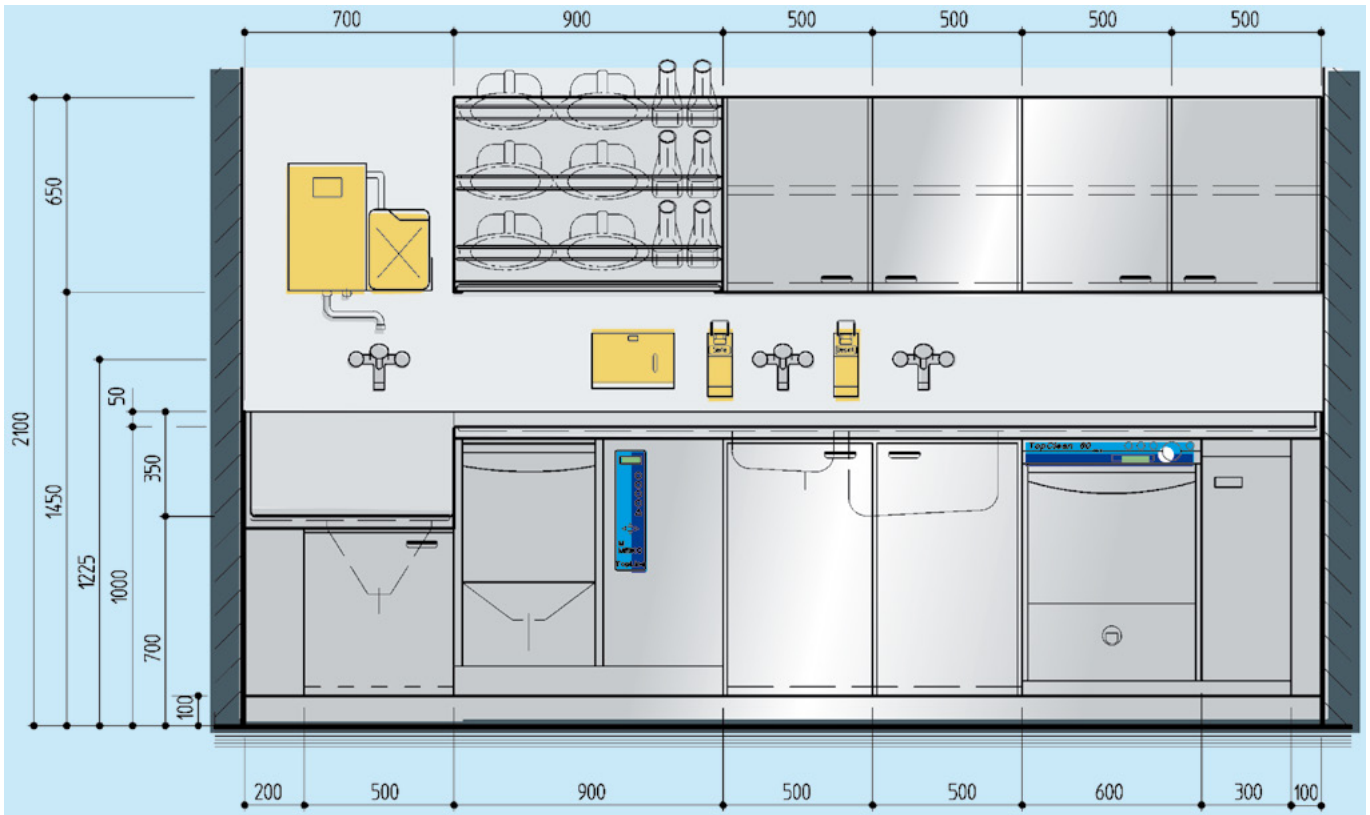


MEIKO TOPIC floor-standing unit

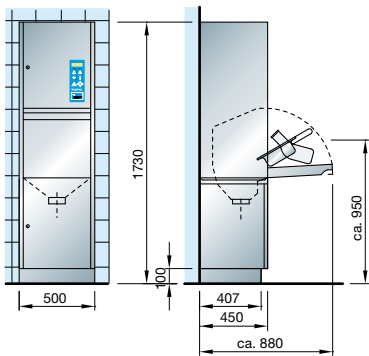

MEIKO TOPIC small combined care unit

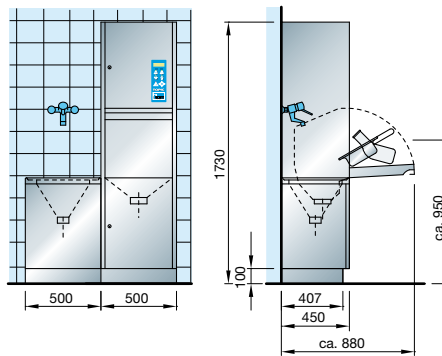


MEIKO TOPIC cabinet model

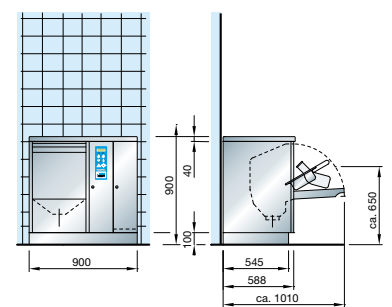

		TOPIC	
		400 V	230 V
Total connected load	kW	4.6	2.8
Feed water connection, cold water		G ¾"	
Feed water connection, hot water 45-60 °C		G ¾"	
Flow pressure	bar	1-5	
Flow rate	l/min	18	
Drain		DN 100	
Noise level	dB(A)	48.2	



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The clean solution



Manufacturer:
MEIKO Maschinenbau GmbH & Co. KG · Englerstr. 3 · 77652 Offenburg, Germany
Phone +49 781 203-0 · info@meiko-global.com · www.meiko.com

760.125.04.06.26/EN/

We reserve the right to amend specifications
as part of our product improvement process